ROLLIT

Inspired by life

Voorwoord

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The Rollit is a comfortable and sturdy wheelchair for permanent use. A solid base frame also makes the Rollit suitable for intensive use. This practical “no nonsense” wheelchair combines extreme durability with superb comfort seating and riding behaviour.

The Rollit is easy to use for both user and attendant. The wheelchair can be adjusted for a specific user by using additional parts and accessories.

The Rollit is resistant to water splashes and is suitable for indoor and outdoor use.

The maximum occupant weight is 160 kg.

The Rollit has been tested positively in compliance with the relevant requirements of the European wheelchair standards EN 12182 and EN 12183.

The Rollit therefore complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices.

The Rollit has not been tested in accordance with ISO 7176-19. For this reason, Life & Mobility recommends that the wheelchair does not have an occupant when it is transported by taxi.

Attention:

- *Read this user manual carefully before using the wheelchair.*
- *When you see this symbol in the manual, it indicates there is an additional instruction. Always follow these instructions. Negligence may adversely affect the operation of the wheelchair and may cause physical injury or damage to the wheelchair or the environment.*

Table of Contents

Safety instructions	5
Checking the delivery	9
Technical specifications	10
Product description	11
Options	12
Settings and adjustments	14
Use	16
Maintenance	18
Technical problems	19
Guarantee	20
Disposal of product and packaging	21

Safety instructions

As the user of the wheelchair, you are always responsible for complying with the applicable safety instructions and guidelines.

Life & Mobility accepts no liability for any loss or injury caused by negligence or by failure to comply with the safety instructions.

Always notify Life & Mobility and the relevant authority in case of a serious incident.

General

Warning:

- Never change the construction of the wheelchair.
- Keep the wheelchair away from open fire.

Attention:

- This wheelchair is suitable for transporting a single person.
- Only use original Life & Mobility parts and accessories.
- Never cover or remove the safety instructions and stickers.

Personal requirements

Attention:

- Your vision must be good enough to ride the wheelchair safely in any possible situation.

Assembly and adjustment

(see also the chapter 'Settings and adjustments')

Warning:

- Before using the wheelchair, check that all its parts are properly mounted and secured.

Safety instructions

Sitting in the wheelchair

Warning:

- *Make sure the fingers and limbs of the person in the wheelchair do not get trapped. Always be very careful:*
 - *When the occupant is sitting lopsided in the wheelchair.*
 - *When the occupant's arms hang limply over the armrest.*
 - *When both of the above occur.*
- *An incorrect sitting posture can cause points of pressure. If you are uncertain about this then immediately contact your doctor or therapist.*

Driving the wheelchair

Attention:

- *When driving up a slope, the centre of gravity will shift and the wheelchair can tip backwards more easily.*
- *Pay attention to your hands when using the handrims if there is not much room at the side(s) of the wheelchair.*
- *Assistants must never leave the wheelchair on a slope without applying the brakes.*
- *Never tackle obstacles higher than 10 cm on your own. Only practised wheelchair riders that have sufficient strength are usually able to tackle obstacles higher than 10 cm.*
- *Set the brake on the wheelchair before getting into or out of the wheelchair.*
- *Only use the brake to lock the wheels when standing still. The brake is not suitable for braking the wheelchair.*

Transporting the wheelchair

Extremely dangerous:

- *The wheelchair is not Crash Test Safe, which means it is not suitable transportation in a taxi when the wheelchair is carrying an occupant.*

Safety instructions


- When transporting the wheelchair, always make sure you are seated in a fixed seat of the motor vehicle.

Attention:

- Only lift the wheelchair using fixed parts of the frame; do not use the removable accessories such as armrests, footrests, etc.

Slopes and other obstacles

(see also page 17 for more information)

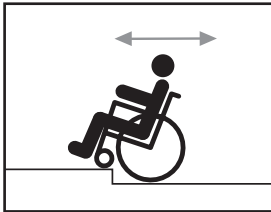Steps

Attention:

- The risk of the wheelchair tipping over on a slope is greater than on a flat surface.
- Always reverse backwards down a step so that the wheelchair cannot tip forward and let you fall.
- Never ride the wheelchair on stairs by yourself.
- Never ride on a slope with an angle of more than 5 degrees.
- Ask for help from bystanders when you are not sure that you can reach the highest point by yourself.
- The attendant pushing the wheelchair must continue to push the wheelchair to prevent it running away.
- Do not turn around on a steep slope. If the wheelchair is at the diagonal on a steep slope, it will significantly increase the risk of tipping.
- When going down a slope, always keep your speed under control.
- The assistant must brake the wheelchair using the push bars.
- When moving the wheelchair by yourself, only use your hands on the handrims to brake and do not use the tyre brake. Otherwise, you risk losing control of the wheelchair.

Slope

Safety instructions

Miscellaneous

Warning:

- *Do not put the wheelchair in direct sunlight: the upholstery can overheat and may cause burns or irritation to the skin.*

Be careful:

- *Do not allow the wheelchair to come in contact with seawater as it is corrosive and will rust the wheelchair.*
- *Do not allow the wheelchair to come in contact with sand, as it may penetrate into the moving parts of the wheelchair, causing unnecessarily fast wear and tear.*


Checking the delivery

Checking the delivery

Before using the wheelchair, check that the delivery matches the product description on page 11. If you find any transport damage on delivery, please inform your dealer. Life & Mobility trusts that the wheelchair meets your expectations and hopes it brings you much convenience.

Technical specifications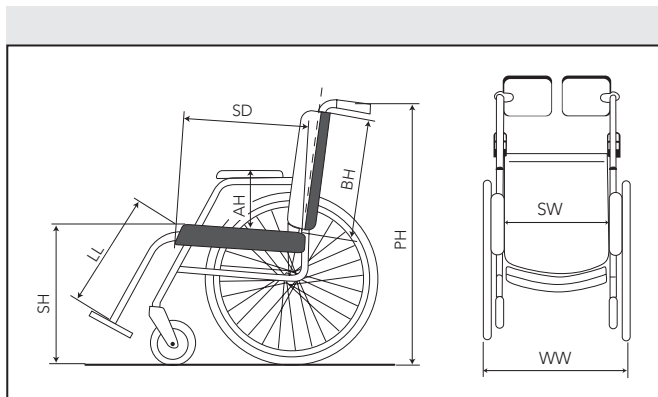

- Seat width (SW): 47 - 52 cm
- Seat height (SH): 51 - 56 cm
- Seat depth (SD) 46 cm
- Back height (BH): 47 cm
- Armrest height (AH): 24 cm
- Wheelchair width (WW): 72 cm
- Push handle height (PH): 92 - 97 cm
- Lower leg length (LL): 45 - 55 cm
- Total weight: 39 kg
- Maximum occupant weight: 160 kg

Life & Mobility recommends using the wheelchair at an ambient temperature between -5°C and +40°C.

Product description

Product description

1. Push bar
2. Padded backrest
3. Handrim (semi-solid rear wheel)
4. Foldable side with padded armrest
5. Padded seat
6. Extended handbrake
7. Tipping lever
8. Front wheel (semi-solid)
9. Foot support + heel strap

EN11

Options

Options (not illustrated)

In addition to the standard parts, the following options are available for the wheelchair:

- Sides
 - Foldable sides with height adjustable armrests
 - Fastening set for sides
- Worktop
 - Perspex worktop
 - Wooden worktop
 - College worktop
 - Elbow padding
- Seat
 - Plank seat (with lip)
 - Contour+ seat
- Backrest
 - Anatomical backrest
- Leg rests
 - Comfort legrest (with heel strap)
 - Knee cushion
 - Plaster cast legrest
 - Amputation support
- Headrest
 - Standard headrest small

Options

- Accessories (not illustrated)
- Catheter hook
- Oxygen bottle holder 12.5 cm
- Crutch holder
- Anti-tipping supports
- Spoke protectors with printed logo
- Infusion pole, height adjustable
- Anti-theft pole

EN13

Settings and adjustments

The wheelchair can be adjusted in various way to meet your particular needs. The adjustments are easy to carry out by yourself.

Attention:

- Check whether the sides are secured in the holders and not to the sides. Otherwise, it may damage the sides and wheels.

Fold up the sides

Grasp and lift the front of the armrest and fold the entire side to the back.

Removing the leg rests

The leg rests are easy to remove from the frame. Grasp the section it is hung on and remove with an upwards motion.

Swinging the leg rests out

1. Lift the leg rest up a little.
2. Swing the leg rest to the side.
3. Lower the leg rest again.

Attention:

- Make sure the foot plates are folded up before you get out of the wheelchair. Never stand on the foot plates. Otherwise, you risk toppling forward in the wheelchair.
- Never use the foot plates to step into the wheelchair.


Lower leg length adjustment

1. Unscrew the socket screw or star grip (A) by one turn.
2. Set the desired lower leg length.


Attention:

- Allow for a distance of at least 5 cm to the floor.
3. Tighten the socket screw or star grip (A).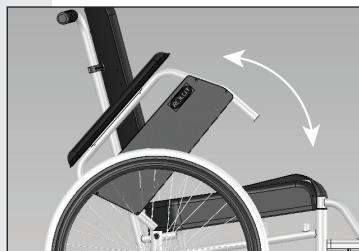


Fold up the sides

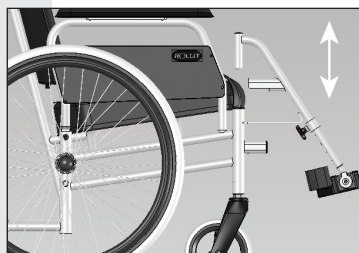Removing the leg rests


Swinging the leg rests out

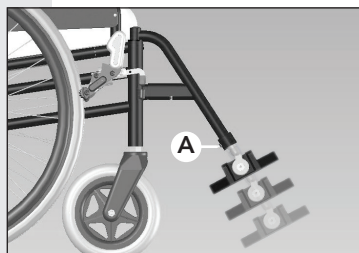Lower leg length

Settings and adjustments

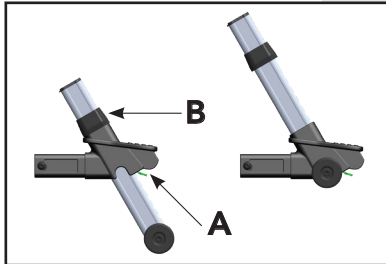Anti-tipping supports (option)

The anti-tipping supports can be adjusted using the lever (A). The recommended height is 3 cm from the floor.

Please note:

- Make sure that the pin clicks properly into the opening.
- Inform the user if the anti-tip support is folded away.

Tip:

- The sliding sleeve can be used to mark a standard position.

The anti-tip support - if properly adjusted - protects the wheelchair user from tipping backwards. The risk of tipping the wheelchair backwards is influenced by:

- the position of the user's center of gravity in the wheelchair (which is determined by factors such as axle position, backrest angle, seat angle, seat tilt, and weight distribution over the body (e.g., amputations)).
- any restlessness or spasms of the user.
- the slope on which you are driving.

Adjust the anti-tip as necessary based on the above considerations.

Use

Removing and fitting the headrest*

The headrest can be removed easily:

1. Unscrew the star grip until loose.
2. Remove the headrest.

Putting it back in place:

3. Place the headrest in the holder.
4. Screw fast the star grip.

* *If present*

Headrest

Making transfers

The Rollit is a wheelchair for users with various limitations. Transfers from the wheelchair to the bed, toilet or shower, and so on, and vice versa, can be carried out in different ways.

The lifting method depends on the degree to which the wheelchair user can cooperate. The wheelchair enables both active and passive transfers with or without the help of an assistant or a lift.

Attention:


- *Always set the brake on the wheelchair before making a transfer.*

Transfer from the front


1. Fold up the foot plates. Or, if necessary, swing out the foot rests.
2. Engage the wheelchair brake.

Transfer from the side

If necessary, fold the sides fitted with arm rests up and backwards.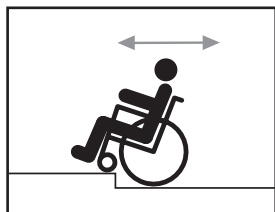

Steps

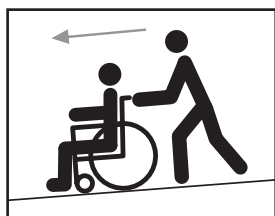Slope

Negotiating obstacles with an attendant

1. Push the step tube down with your foot and push the handles down with your hands.
2. Push the wheelchair onto the elevation (obstacle) and take your foot off the step tube.
3. Push the wheelchair further and lift the rear wheels over the obstacle with the push handles.

Go up steps forward and down steps backward.

Attention:

- *Keep speed under control when going down a gentle slope.*

Brakes

The wheelchair brakes are adjusted to the proper setting at the factory as standard. Life & Mobility recommends that you always use both brakes when getting in or out of the wheelchair.

Maintenance

Minor maintenance (every 2 weeks)

You can perform minor maintenance on the Rollit yourself.

Cleaning

Clean your wheelchair with a damp cloth, water and multi-purpose cleaner.

Attention:

- *Never use abrasive or harsh cleaning products or solvents, such as thinner, benzine (a.k.a. white spirit), etc.*

Check that all fastening parts are properly fastened. Re-tighten them if necessary.

Attention:

- *Tightening the adjustment screw too much can damage the wheelchair.*

Disinfecting

You may use alcohol (70%) or chlorine (1,000 ppm) to disinfect the wheelchair.

Wheeltension

Check the wheel tension on a regular basis (once per quarter, depending on the intensity of use) to make sure that your wheelchair continues to ride comfortably and safely. Ask your dealer for assistance if necessary.

Major maintenance (annual)

Your dealer would be happy to handle the major maintenance for your wheelchair. That way you are certain you are getting the best possible use from your wheelchair. You can also go to your dealer for parts replacement.

Technical problems

Should your wheelchair be subject to unexpected technical problems, we advise you to contact your dealer. The dealer will assess whether he can repair the chair or whether the wheelchair has to be returned to the manufacturer for repairs.

You may be able to solve a number of problems yourself.

Moving parts, such as, arm rest, foot/leg rests, and sides will not move:

- The above-mentioned parts may have become stuck in the frame when excessive force was used on these parts. Carefully tap the components using a plastic hammer.

The brakes do not have sufficient braking power or the brakes on either side do not have the same braking power:

- You must have the brakes adjusted. Consult your dealer for this task.

Attention:

- *All repairs other than those mentioned above always should be carried out by a certified dealer. Careless repairs can endanger your health. Moreover, the warranty on the product will be cancelled. Use common sense and decide whether you yourself are capable of solving technical problems of the chair. Always consult your dealer if you have any doubt. When fitting and replacing parts, you must comply with this manual and your dealer's advice. Prevent any hazardous situation occurring for the assistant and the occupant of the chair!*

Guarantee

Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product.

Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated verbally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

Disposal of product and packaging

Disposal of product and packaging

From an environmental perspective, we recommend that you dispose of the packaging and the wheelchair by separating the waste as much as possible, allowing the waste materials to be recycled efficiently.

In most cases you can hand in your used wheelchair to your wheelchair dealer. If this is not possible, you can hand in the wheelchair at your local waste collection station.

This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication.

The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.

The information in this manual may not be reproduced for commercial goals and/or made public by any means without prior written permission from Life & Mobility.

Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.

No rights can be derived from this text. Subject to changes.

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© Life & Mobility – All rights reserved