MEZZO

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Dear user,

By choosing a Mezzo mobility scooter you have chosen unequalled security, quality and driving pleasure! We hope you will drive many comfortable and safe miles and trust that you will enjoy the freedom offered by the Mezzo.

In this user manual you will find everything you need to know about the Mezzo, from safety instructions and operating functions to maintenance tips. Carefully read the user manual, so you can get the most out of your mobility scooter. Make sure that you do not lose the user manual for a possible future user.

We would like to keep you informed about updates, user experiences and other useful information. On the website **[www.life-mobility.com](http://www.life-mobility.com)** you can regularly read about the latest developments.

Kind regards,  
Management and staff of Life & Mobility

## Contents

Safety instructions	5
Introduction	11
Technical specifications	12
Product description	16
Settings and adjustments	21
Use	24
Maintenance	31
Transport and storage	35
Faults	37
Default settings	38
Disposal of product and packaging	39

## Safety instructions

As the user of the mobility scooter, you are always responsible for complying with the locally applicable safety instructions and guidelines. It is important to first acquire driving experience under the supervision of an experienced instructor. Your dealer can advise you on this.

Life & Mobility accepts no liability for any loss or injury caused by negligence or by failure to comply with the safety instructions.

**Attention:**

- If you see this symbol in the manual, it indicates a supplementary instruction. Always follow this instruction. Negligence may adversely affect the operation of the mobility scooter and may cause physical injury or damage to the mobility scooter or the environment.

**General**

**Warning:**

- Never change the construction of the mobility scooter.
- Keep the mobility scooter away from open fire.

**Be careful:**

- Use the basket only to transport objects.
- Do not use your mobility scooter to pull vehicles, carts, etc.
- Remember that there are protruding parts, both fixed and loose, in and on the mobility scooter. These can cause damage to the surroundings and the mobility scooter.

**Attention:**

- This mobility scooter is suitable for transporting a single person.
- Only use original Life & Mobility parts and accessories.
- The mobility scooter has a fully programmable electronic control system. Changes to the programming must only be made by Life & Mobility.
- Never cover or remove the safety instructions and stickers.
- Always switch off the mobility scooter and take out the ignition key before leaving it. This prevents unwanted use and/or theft.

**Personal requirements****Warning:**

- If you are under the influence of substances that may affect your ability to drive, you may not drive the mobility scooter.

**Attention:**

- Your vision must be adequate to drive the mobility scooter safely in any situation that may occur during use.
- Please follow the relevant traffic rules that apply.
- Make sure that you can always be seen by other road users. As a user of a mobility scooter you are very vulnerable on the road.
- Avoid quiet routes. Make sure that help can be arranged in the event of an emergency.
- Switch off the mobility scooter if you are using your mobile telephone. Your mobility scooter may activate security gates in shops.

**Assembling and adjustment**

(see also the chapter 'Settings and adjustments')

**Warning:**

- Before using the mobility scooter, check that all components of the mobility scooter are properly mounted and secured.

**Attention:**

- Your dealer can set up the various driving features for you with the microprocessor.

**Sitting in the mobility scooter****Warning:**

- Make sure that your fingers and limbs do not get trapped. Always be very careful:
  - When folding away armrests.
  - If you are not in full control of your arms or legs.

**Driving the mobility scooter**

(see also the chapter 'Use')

**Warning:**

- Drive carefully on roads that are slippery due to rain, black ice or snow.
- Make sure that no loose clothing can get caught. It can obstruct the wheels and cause the scooter to overturn.
- Watch out for overturning:
  - If you have set up the seat with a large back angle.
  - When driving up a slope.
  - If the slide system is in the rear-most position.
- Do not transport anything on the handlebars.
- Do not let anyone stand on the rear of your mobility scooter.

### Slopes and other obstacles

#### Warning:

- Avoid slopes with a gradient greater than 10°. Driving on slopes (forwards or in reverse) is always dangerous.
- You must never free wheel down a slope.
- Drive carefully along (pavement) edges. An uneven surface increases the risk of overturning.
- Avoid prolonged driving on inclines <math><10^\circ</math>. This can cause excessive wear on the engine and electrical components.

Maximum gradient

#### Attention:

- Your centre of gravity moves when you drive up or down a slope. Please beware.
- Turn around carefully on a steep slope as there is a greater risk of overturning.
- Always mount and leave a pavement in a forward, straight line. Accelerate very gently. It is better to mount the pavement using a curb cut.
- The maximum bump height you can drive over is 10 cm.
- Soft, slippery or uneven surfaces increase the risk of a dangerous situation.

a forward, straight line

Approaching a pavement

### Transporting the mobility scooter without a driver

(see also the chapter 'Transport and storage')

#### Life-threatening:

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.

#### Warning:

- Make sure that your fingers do not get trapped when folding down the mobility scooter.
- Never carry the mobility scooter by the plastic parts of the housing.

#### Attention:

- Make sure that the mobility scooter is fully assembled before use.

### Miscellaneous

#### Warning:

- Do not put the mobility scooter in direct sunlight: the upholstery can become too hot, and cause burns or irritation to the skin.

#### Be careful:

- Do not allow the mobility scooter to come into contact with seawater: seawater is corrosive and will damage the mobility scooter.
- Do not allow the mobility scooter to come into contact with sand: sand may penetrate to the moving parts of the mobility scooter, causing faster wear and tear.

#### Attention:

A number of technical precautions have been made to further guarantee your safety:

- The immobiliser is activated whenever a battery charger is connected.
- The speed in bends is automatically limited.
- Free wheeling is limited.
- The display warns you if the battery voltage is too low.
- The display gives you information for resolving possible faults.
- In the event of an electronic fault, the mobility scooter make a controlled stop.

## Introduction

- *The control panel of the mobility scooter includes a stop button to stop the scooter quicker in an emergency.*
- *The mobility scooter has ergonomically shaped safety handlebars to prevent injuries to fingers and hands.*
- *To avoid the risk of parts of the body becoming jammed, the necessary components have been shielded.*
- *Your dealer can fit a seatbelt to the fastening points of the armrests that are attached to the backrest, to secure your body in the correct position.*

### **EMC-approved**

The mobility scooter is approved for EMC (Electro-Magnetic Compatibility). Tests have shown that mobility scooters are not sensitive or damaging to mobile communication equipment and electronically controlled equipment. Despite this, please be careful when using the mobility scooter and mobile or electronic equipment at the same time.

The Mezzo is an electrically powered mobility scooter, suitable for both young and older people who have difficulty walking long distances.

You can set up the Mezzo mobility scooter to meet all your requirements. This guarantees a relaxed posture while driving. The mobility scooter is basically operated with two hands. You need not be strong to drive the mobility scooter and the controls are easy. The Mezzo is available as a 3-wheel and 4-wheel scooter.

This manual describes the default settings of your mobility scooter. In the back of this user manual a list has been included for recording your specific settings.

## Technical specifications Mezzo 3

- Intended use: outdoors, between -25° C and + 50° C, on paved paths and roads
- L x w x h: 138 x 64 x 119 cm
- L x w x h (without seat): 138 x 64 x 88 cm
- Weight without batteries (*with batteries 60 Ah*): 80 kg (*118 kg*)
- Weight of heaviest removable part (seat): 15 kg
- Maximum static stability downhill: 15 °
- Maximum static stability uphill: 15 °
- Maximum static stability sideways uphill: 15 °
- Maximum dynamic stability uphill: 10 °
- Maximum safe gradient: 10 °
- Max. driving distance with fully-charged batteries\*: 35 km (22 miles - 50 Ah) / 40 km (25 miles - 60 Ah) / 50 km (31 miles - 73 Ah)
- Maximum obstacle height: 10 cm
- Maximum obstacle drop (pavement): 10 cm
- Ground clearance: 8 cm
- Max. (programmable) speed: 15 km/h\*\*
- Minimum braking distance (programmable): 300 cm
- Seat angle: 5 °
- Seat depth: 42 cm
- Seat width: 48 cm
- Seat height above footrest (min. - max.): 44 - 54 cm
- Height of footrest above ground: 17 cm
- Back angle: 90 - 180°
- Back height: 48 cm
- Armrest height (to seat): 20 cm
- Armrest length (to backrest): 26 cm
- Turning circle (radius): 140 cm
- Turn around width: 160 cm
- Test dummy weight: 160 kg
- Max. user weight: 160 kg
- Required operating force: approx. 1 N
- Battery capacity (optional): 50/60\*\*\*/73\*\*\* Ah
- Max. load: 8 amps
- Motor, transaxle: 500 watts 24 V
- Magnetic brake: 8 Nm 24 V

\* *Depending on the set speed, outside temperature, weight of the user, tyre pressure, etc.*

\*\* *Less is possible*

\*\*\* *Gel batteries*

- Ability to dismantle for transport: limited
- Wheel size: 3.00 – 8
- Complies with: EN 12184 class C

The mobility scooter has 2 12-volt dry (gel) batteries.

The following types are suitable:

- 50 Ah (228 x 140 x 255 mm)
- 60 Ah\*\*\* (259 x 169 x 178 mm)
- 73 Ah\*\*\* (276 x 171 x 210 mm)

### Test specifications

- The scooter conforms to the following standards: ISO 7176- 8,9,14,16
- Requirements and test methods for static, impact and fatigue strengths: ISO 7176-8
- Climatic tests for scooters: ISO 7176-9
- Power and control systems for scooters: ISO 7176-14
- Requirements for resistance to ignition: ISO 7176-16

### Service and technical support

Contact your dealer if you have any questions about specific settings, maintenance or repairs. Always have the serial number ready. You can find these on the mobility scooter's CE sticker (see also page 20).

### Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

## Technical specifications Mezzo 4

- Intended use: outdoors, between -25° C and + 50° C, on paved paths and roads
- L x w x h: 138 x 64 x 119 cm
- L x w x h (without seat): 138 x 64 x 65 cm
- Weight without batteries (*with batteries 60 Ah*): 89 kg (127 kg)
- Weight of heaviest removable part (seat): 15 kg
- Maximum static stability downhill: 15 °
- Maximum static stability uphill: 15 °
- Maximum static stability sideways uphill: 15 °
- Maximum dynamic stability uphill: 10 °
- Maximum safe gradient: 10 °
- Max. driving distance with fully-charged batteries\*: (30 km) 19 miles (50 Ah) / (35 km) 22 miles (60 Ah) / (45 km) 28 miles (73 Ah)
- Maximum obstacle height: 10 cm
- Maximum obstacle drop (pavement): 10 cm
- Ground clearance: 8 cm
- Max. (programmable) speed: 15 km/h\*\*
- Minimum braking distance (programmable): 300 cm
- Seat angle: 5 °
- Seat depth: 42 cm
- Seat width: 48 cm
- Seat height above footrest (min. - max.): 44 - 54 cm
- Height of footrest above ground: 17 cm
- Back angle: 90 - 180°
- Back height: 48 cm
- Armrest height (to seat): 20 cm
- Armrest length (to backrest): 26 cm
- Turning circle (radius): 167 cm
- Turn around width: 195 cm
- Test dummy weight: 160 kg
- Max. user weight: 160 kg
- Required operating force: approx. 1 N
- Battery capacity (optional): 50/60\*\*\*/73\*\*\* Ah
- Max. load: 8 amps
- Motor, transaxle: 500 watts 24 V
- Magnetic brake: 8 Nm 24 V
- Ability to dismantle for transport: limited

\* Depending on the set speed, outside temperature, weight of the user, tyre pressure, etc.

\*\* Less is possible

\*\*\* Gel batteries

- Wheel size: 3.00 – 8
- Complies with: EN 12184 class C

The mobility scooter has 2 12-volt dry (gel) batteries. The following types are suitable:

- 50 Ah (228 x 140 x 255 mm)
- 60 Ah\*\*\* (259 x 169 x 178 mm)
- 73 Ah\*\*\* (276 x 171 x 210 mm)

### Test specifications

- The scooter conforms to the following standards: ISO 7176- 8,9,14,16
- Requirements and test methods for static, impact and fatigue strengths: ISO 7176-8
- Climatic tests for scooters: ISO 7176-9
- Power and control systems for scooters: ISO 7176-14
- Requirements for resistance to ignition: ISO 7176-16

### Service and technical support

Contact your dealer if you have any questions about specific settings, maintenance or repairs. Always have the serial number ready. You can find these on the mobility scooter's CE sticker (see also page 20).

### Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

## Product description Mezzo 3


### Checking the delivery

Before using the mobility scooter, check if the delivery matches the following product description. If you find any transport damage upon delivery, please inform your dealer. Life & Mobility trusts that the mobility scooter meets your expectations and hopes it brings you much convenience.

### Product description

#### Mobility scooter


- A. Handlebars with control panel
- B. Basket
- C. Headlight
- D1. Indicators (orange)
- D2. Brake lights / Rear light (red)
- E. Front wheel
- F. Footrest
- G. Battery cover
- H. Freewheel lever
- I. Rear wheel
- J. Electronics cover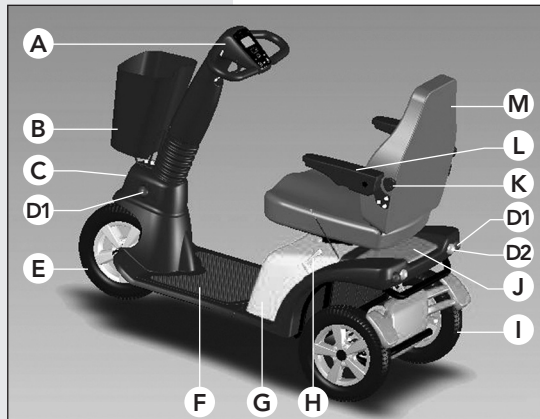

Mobility scooter product description

#### Seat

- K. Lumbar adjustment button
- L. Folding armrests
- M. Seat
- N. Armrest adjusting wheel
- O. Back angle lever
- P. Seat lever  
(up/down and rotate)

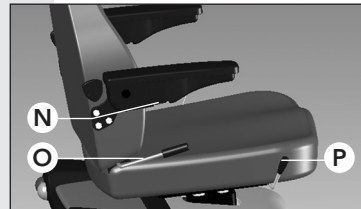Product description seat

#### Handlebars

- Q. Ignition lock
- R. Handlebar height socket screw
- S. Battery charger contact
- T. Handlebar angle adjustment lever

Product description handlebars

## Product description Mezzo 4


### Checking the delivery

Before using the mobility scooter, check if the delivery matches the following product description. If you find any transport damage upon delivery, please inform your dealer. Life & Mobility trusts that the mobility scooter meets your expectations and hopes it brings you much convenience.

### Product description

#### Mobility scooter


- A. Handlebars with control panel
- B. Basket
- C. Headlight
- D1. Indicators (orange)
- D2. Brake lights / Rear light (red)
- E. Front wheels
- F. Footrest
- G. Battery cover
- H. Freewheel lever
- I. Rear wheel
- J. Electronics cover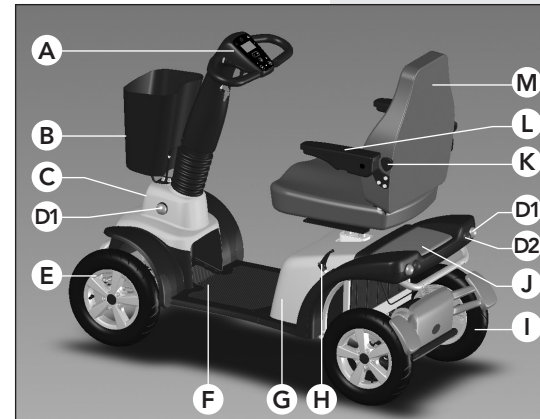

Mobility scooter product description

#### Seat

- K. Lumbar adjustment button
- L. Folding armrests
- M. Seat
- N. Armrest adjusting wheel
- O. Back angle lever
- P. Seat lever  
(up/down and rotate)

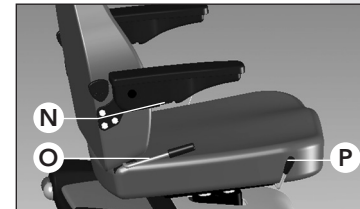Product description seat

#### Handlebars

- Q. Ignition lock
- R. Handlebar height socket screw
- S. Battery charger contact
- T. Handlebar angle adjustment lever


Product description handlebars

## Product description

### Control panel

- U. Stand-by button / Stop button
- V. Warning lights
- W. Display
- X. Speed decrease button
- Y. Forwards / reverse throttle lever\*
- Z. Reverse button or Hi Lo function
- a. Right indicator
- b. Left indicator
- c. Horn
- d. Right indicator
- e. Left indicator
- f. Forwards / reverse throttle lever\*
- g. Speed increase button
- h. Menu button
- i. Lights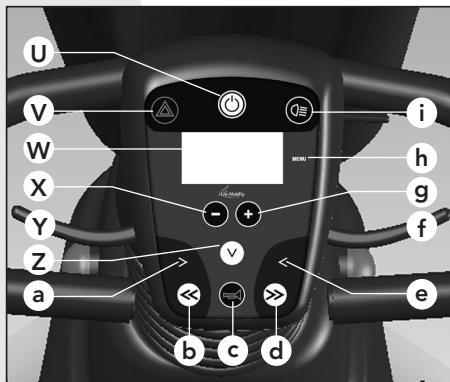

Product description  
control panel

### Features

- Basket
- Lights
- Brake lights
- Horn
- Torsion springs on front and rear wheels
- Semi-independent suspension on the rear axle (and on the front axle of 4-wheelers)
- Comfortable seat with preshaped seating surface and backrest with adjustable lumbar support
- Easy and light operation of handlebars and control levers
- Shown on the display: km/h, journey distance, total distance (km), date and time
- Clear battery display and low battery voltage warning
- On/off button for (de)activating driving mode

\* Depending on how the mobility scooter is set up.

### Safety and comfort features

- Ergonomic safety handlebars with soft padding and height and depth adjustment via a gas spring
- Large headlight
- Limited freewheel (4 km/h)
- Automatic speed limiter in bends
- Indicators cancel automatically after turn
- Mirror (optional)
- Footpedal accelerator (optional)

### Not pictured

Various accessories are optionally available, including:

- Adjustable armrests
- Soft filling armrests
- Slide system under chair
- Seatbelt
- Extra footpedal accelerator incl. lever control
- Rearview mirror
- Anti-tipping supports
- Eiffel crutch holder
- Rollator holder
- Rearside bar on chair for extra basket
- Extra basket Cardiff
- Scoox set-in bag for Cardiff basket
- Foot support
- Lined scooter cape, rain and wind proof
- Rainproof scooter cover
- Adapter for Goliath 3 / Hercules tackle system
- 4 Fixation points (Mezzo 3)/ 2 Fixation points (Mezzo 4)

See [www.life-mobility.com](http://www.life-mobility.com) for more accessories.

\* Not possible in conjunction with Hi Lo function.

## Settings and adjustments


### Markings on the mobility scooter

There are a number of stickers on the mobility scooter. Below is an overview of these stickers, with their location.


#### CE sticker

The mobility scooter complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices and has a CE label.

The CE sticker is located underneath the electronics cover (J) at the rear of your mobility scooter. Slide the cover backwards and place this separately.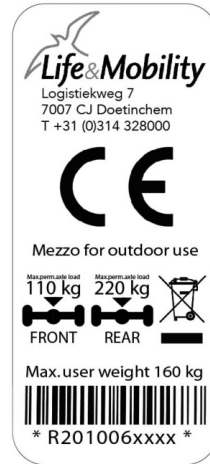

CE sticker

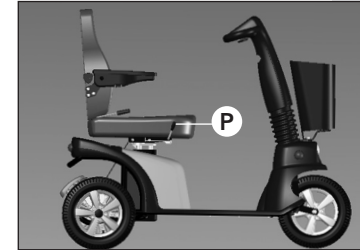Change seat height

You can set up the mobility scooter to meet all your requirements. It is advisable to contact your dealer for the correct settings.

### Adjusting the seat

#### Seat height

1. Move the lever (P) upwards (it is located on the left-hand or right-hand side of the seat, depending on the configuration chosen).  
The seat will go down if you are sitting in it.  
The seat will go up if you are standing next to the mobility scooter.
2. Release the lever (P).  
The seat is now locked in its current position.

#### Back angle


#### Attention:

- Only adjust the backrest when you are sitting in the seat.
- The maximum back angle (180°) is only intended for transporting the seat.
- Watch out for overturning:
  - If you have set up the seat with a large back angle.
  - When driving up a slope.
  - If the slide system is in the rear-most position.

#### Attention:

- Always slide the electronics cover forward again, as it protects the electronic equipment underneath.
- You will hear a click when the electronics cover is in the correct position.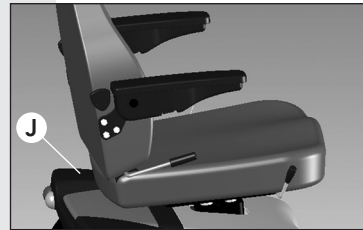

#### Freewheel sticker (see also page 29)

The sticker above the freewheel lever indicates the "push position". In this setting the mobility scooter can be moved freely.

The sticker below the free-wheel lever indicates that the mobility scooter is ready to be electrically driven.

#### Battery connection diagram

The battery connection diagram is located on the inside of the battery cover. It also includes the fuse and the motor controls.

Push position

Electric driving position

Change back angle

1. Sit down on the seat.
2. Lift up the lever (O).
3. Use your upper body to push the backrest backwards into the right position, or:
4. Bend your upper body forward to allow the backrest to move forward automatically.
5. Release the lever (O).  
The backrest is now locked in its current position.


**Armrests**

**Attention:**

- The position of the armrests changes if the position of the backrest is adjusted.


1. Rotate the armrest upwards to the desired angle.
2. Rotate the adjusting wheel (N) to lock the angle of the armrest.
3. Loosen the adjusting wheel (N) to move the armrest down.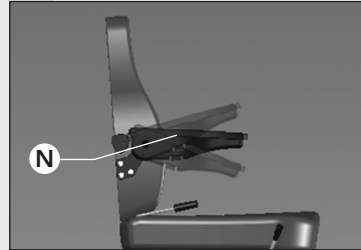

Change armrest angle

**Backrest lumbar support**

1. Turn the button (K) once at a time. This changes the curve of the backrest.
2. Check if the new setting has the desired result. There are 3 positions.

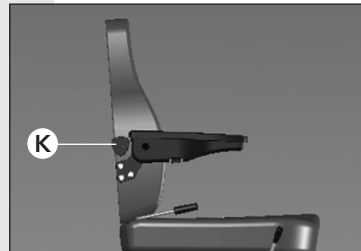Set backrest lumbar support

**Adjusting the handlebars**

**Be careful:**

- Do not pull the handlebars upwards too firmly when adjusting; the maximum setting is 8 cm.

**Attention:**

- Have your dealer set the handlebar height in the correct position once and for all before using the mobility scooter.
- Do not adjust the angle of the steering column while driving. A gas spring allows the handlebars to move forward independently.

**Handlebar height**

1. Loosen the (socket) screw (R) using an Allen key.
2. Adjust the height of the handlebars.
3. Tighten the screw again firmly.

Set handlebar height

Change handlebar angle

**Steering column angle**

1. Sit down on the seat.
2. Squeeze the lever (T)\*.
3. Pull the steering column towards you or push it away until you reach the desired position.
4. Release the lever.

The steering column is now locked in place.

\* It is located on the left or right-hand side, depending on the configuration of your mobility scooter.

# Use

## Advance checks

Check the following before driving off:

- Whether the lights are working,
- The battery voltage on the display (see page 28),
- The tyre pressure (see the chapter 'Maintenance'),
- Whether the freewheel lever is pointing down (electric driving position, see page 20),
- Whether the battery charger has been removed.

## Getting in and out by yourself

### Attention:

- Make sure that the seat is properly locked in place before driving off.

### Getting in/out

1. Push the seat lever (P)\* down.
2. Rotate the seat a quarter turn anticlockwise or clockwise.
3. Get in or out.
4. Rotate the seat back again until you hear a click.

The seat is now locked in place.


### Starting

1. Insert the key in the ignition (Q).
  2. Turn the key a quarter turn clockwise.
  3. Wait until '0 km/h' appears on the display.
- The mobility scooter is now ready to drive.

## Speed

The speed can be adjusted for either indoor or outdoor use.


\* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.

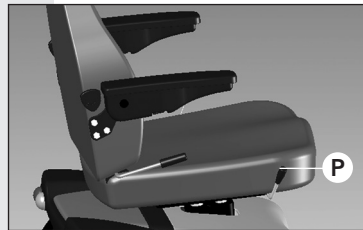Product description seat

Product description handlebars

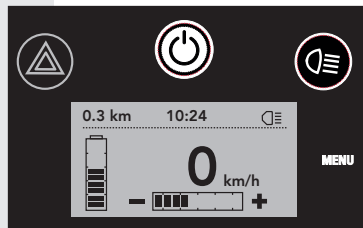Ready for use


### Warning:

- Adjust your speed before steering into a bend. The bend speed limiter will not adjust your speed until the bend is too tight. The scooter may overturn.
- Despite the small turning circle of the mobility scooter, it may still be difficult to ride into lifts or doorways. Make sure that you always use the correct low speed setting for tight manoeuvring.

### Attention:

- Adjust your speed using the throttle lever. The speed settings determine how fast you can go if you squeeze the throttle lever all the way.
- Check that the road behind you is clear when reversing.
- The mobility scooter reverses more slowly than when it drives forward.
- An alarm can be heard when reversing.
- Upon request, your dealer can change the control (method) of the throttle lever.

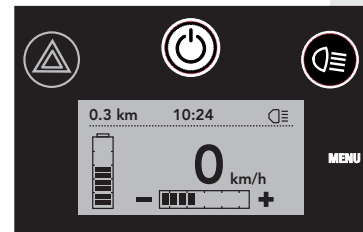Default view speedometer

Indoors, in shops, etc.

On the pavement

In busy streets

On quiet streets

By default the speedometer is set to 4 blocks, which is 40% of the total power. By default the maximum speed is set to 9 mph (15 km/h). Your dealer can adjust the speed setting for you.

1. Press the + or - button to adjust the speed setting.

Choose a suitable speed setting for each situation.


- Indoors, in shops, etc., 1 to 2 blocks
- On the pavement: up to 4 blocks
- In busy streets: up to 7 blocks
- On quiet streets: up to 10 blocks

### Driving forward and reversing

1. Slowly squeeze the throttle lever (f)\*.  
*The mobility scooter starts moving forwards.*
2. Squeeze the throttle lever further.  
*The mobility scooter will start moving forward more quickly.*
3. Release the throttle lever.  
*The mobility scooter will stop. The brake lights are activated.*
4. Use lever (Y)\* to reverse.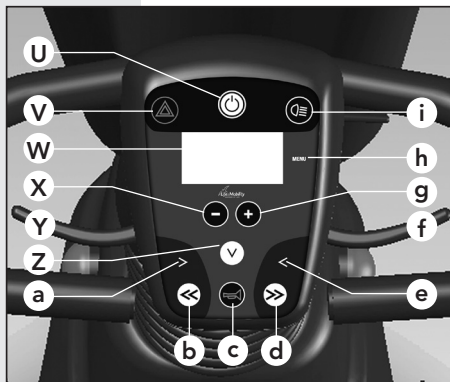


Control panel

### Stand-by button / Stop button

The stand-by button / Stop button (U) has two functions:

- The mobility scooter is temporarily 'switched off'. You can then reach over the handlebars safely without turning the ignition key. The drive controls are switched off, preventing you from driving off unintentionally.
- If you use the stand-by button / stop button while driving, you will come to a standstill more quickly.

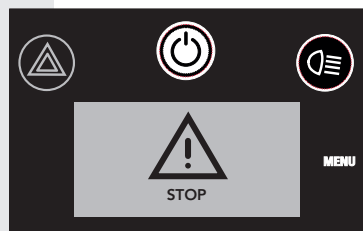Stand-by button / Stop button activated

### Lights

#### Attention:

- After a tight bend the indicators automatically switch off.
- Your dealer can set up one-handed controls for you.
- Your dealer can disable the default sound from the warning lights.

1. Press the (i) button to switch the lights on or off.
2. Press the (b) or (d) button to switch the indicators on or off.

\* Depending on how your mobility scooter has been set up.

3. Press the (V) button to switch the warning lights on or off.

### Batteries

#### Caution:

- New batteries must be run in. The full capacity of batteries is only reached after they have been used and charged a couple of times. Take this into account during your first few trips.
- Charge the batteries every day if you drive a lot, and every few days if you drive little. This will extend the service life of the batteries.
- Always charge the batteries to full capacity.
- Avoid draining the batteries completely. The mobility scooter will lose power which will have a negative effect on the service life of the battery.
- Charge fully drained batteries as soon as possible using a 24-hour cycle.
- Do not keep your batteries fully drained for a prolonged period of time.
- Never put metal parts on the batteries: this may short-circuit the batteries and damage them.

#### Attention:

- At lower temperatures the capacity of the batteries is reduced. This also reduces the operating range.
- Always connect the charger when putting the mobility scooter away for the night.
- You cannot overcharge the batteries supplied. Thanks to a voltage limiter, the charger automatically switches off when the batteries are full.
- Only use Life & Mobility battery chargers.
- If you use batteries other than the batteries supplied by Life & Mobility ensure that a good retaining ring is fitted to prevent them moving. If a retaining ring is not fitted this could lead to short-circuiting.

### Hi Lo function

This function comes as standard in the UK and is optional in other countries.


#### Reading the battery status

The battery-shaped indicator on the display shows the remaining battery capacity:


Battery status	Indicator
All blocks visible	The batteries are full.
3 blocks visible	You can drive another 5 - 10 km (3 - 6 miles) (depending on the type of battery and other circumstances). You should start charging now.
1 block visible	Charge the battery of the mobility scooter as soon as possible. The display shows a warning.

#### Charging batteries


1. Insert the plug of the battery charger into the battery charger contact (S).
2. Insert the plug of the mains cord in a wall socket.
3. Always fully complete the charging cycle of 8 - 10 hours.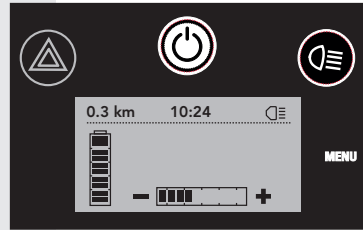

Battery status: full

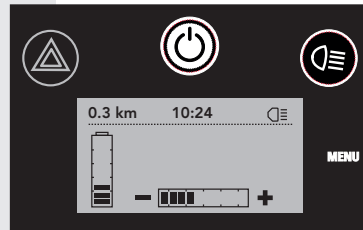Battery status: 3 - 6 miles remaining (5 - 10 km)

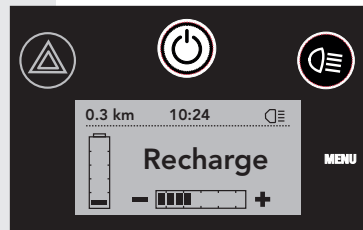Battery status: empty

Battery charger contact


Freewheel lever

### Freewheel lever

#### Warning:

- Never use this function on a slope or while driving. It will disable the brakes. Always turn the freewheel lever downwards to re-engage the brakes after using the freewheel function.


1. Turn the freewheel lever upwards to move the mobility scooter by hand. A beep sounds every 2 seconds to remind you that the freewheel function is engaged.
2. Turn the freewheel lever down for the electric drive position. Teach yourself to do each time you have used it.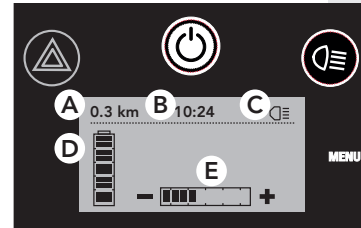


Home screen

### User menu

1. Home screen  
By default the (home) screen of the control panel displays the following information:  
- the distance covered (A),  
- the time (B),  
- lights on/off (C),  
- the battery status (D),  
- the speed (E)
2. Press the menu button.  
The second menu (distances) is displayed and shows the following:  
- the total distance covered (F),  
- the journey distance (G).
3. Press the menu button again.  
The third menu is displayed and shows the following:  
- the time and date (H).
4. Wait a few seconds.  
The screen automatically jumps back to the home screen.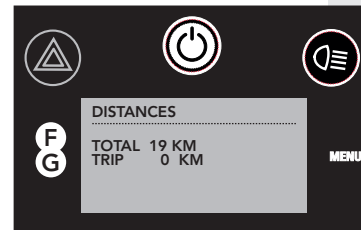

Distances menu

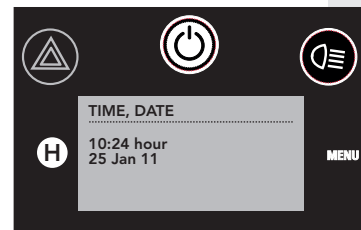Time & date menu


## Maintenance

### Setting the journey counter to 0

1. Hold the menu button down for 2 seconds in the second menu (distances).  
*The journey counter is reset to 0.*

### Setting the time and date

1. Select the third menu (time & date).  
*The 'hours' will start to flash in the edit menu.*
3. Use the + and – buttons to set the correct hour.  
*The minutes will start flashing.*
4. Press the menu button again to save the hours.
5. Use the + and – buttons to set the correct minutes and press the menu button.  
*The date will start flashing.*
6. Use the + and – buttons to set the correct day, month and year.
7. Press the menu button to confirm.  
*You will automatically return to the third menu (time & date).*

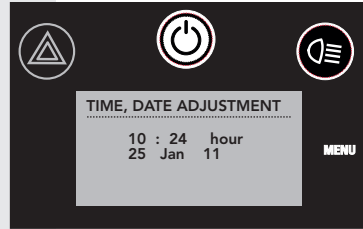Setting the time & date

### Parking

When parking your scooter, place a chain lock through one of the wheels for added security.

### Cleaning

#### **Be careful:**

- Do not use bleach or solvents.
- Do not use a high-pressure cleaner to clean the mobility scooter.

#### **Attention:**

- Regularly clean the upholstery and the frame.

### Upholstery

1. Remove any light dirt with a solution of a mild cleaning agent and warm water.
2. Carefully scrub the upholstery with a soft brush if the dirt has been absorbed into the upholstery.

### Frame

1. Clean the housing with a non-abrasive cleaning product.
2. Apply non-abrasive car wax to the entire frame.

### Batteries (see also the chapter 'Use')

#### **Be careful:**

- Batteries can be dangerous. Only allow your batteries to be installed and maintained by a qualified scooter mechanic. They have the required qualifications and tools to perform the installation and maintenance safely and correctly.
- The scooter uses two batteries with a 24-volt series connection. Never use both batteries at the same time:
  - Batteries from different manufacturers
  - Batteries with different technologies
  - Batteries with different date codes

**Attention:**

- Always fully charge batteries before using them for the first time.
- Charge the batteries continuously for 24 hours at least once a month.
- The battery connection method is illustrated on a diagram on the inside of the battery cover.
- Be careful with damaged batteries. Leave (environmentally-friendly) replacement of the batteries to your dealer.

**Tyres****Warning:**

- For safety reasons always ensure that the tyre pressure on your mobility scooter is correct. If the tyre pressure is too low, your scooter's roadholding will deteriorate, its operating range will decrease and the steering will become heavier.

**Attention:**

- In the event of a flat tyre, please contact your dealer.
- The pneumatic tyres of the mobility scooter must have sufficient pressure. Check this regularly.
- If the tyre pressure is too low you will not be able to drive as far.
- The pressure of the front tyre must be at least 2.0 bar.
- The pressure of the rear tyres must be at least 2.5 bar.
- The maximum tyre pressure is 3.5 bar.
- If the tyre pressure is too high, driving will become uncomfortable.
- Regularly check the tyres for wear and tear (due to drying out).

**Service and technical support**

At least once a year have the mobility scooter checked and serviced by your dealer. Preventive maintenance ensures that the proper and correct functioning of all moving parts is checked, as well as the electronics and the condition of the batteries.

Repairs should only be performed by your dealer or by Life & Mobility, and only original parts supplied by Life & Mobility should be used for this.

## Transport and storage

### Maintenance matrix

The following matrix indicates which parts should be regularly checked. If a defect occurs, contact your dealer to resolve this issue.

Part	Action	Daily	Weekly	Monthly	Each quarter	Annually
Battery charger	Check functioning	X				
Tyres	Check tyre pressure (front: 2.0 bar, rear: 2.5 bar)		X			
Lights	Check functioning	X				
Housing	Clean with damp cloth		X			
Connections	Check for breakages		X			
Rotating mechanism	Check functioning and play			X		
Batteries	24-hour charging cycle			X		
Parking brake	Check functioning				X	
Tyres	Check wear and tear				X	
Mobility scooter	Servicing					X

### Transporting the mobility scooter


#### Extremely dangerous:

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.

#### Be careful:

- Never carry the mobility scooter by the plastic parts of the housing.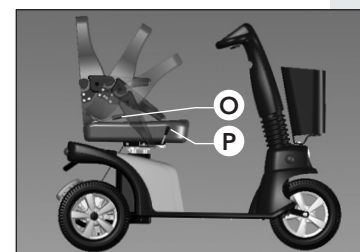

Folding down the backrest

#### Folding down the backrest

- Stand next to the mobility scooter.
- Firmly grip the backrest.
- Lift lever (O).
- Push the backrest forward.
- Release lever (O).

#### Removing the seat

- Turn lever (P) downwards.
- Turn the seat a little.
- Lift the seat from the mobility scooter.

#### Attention:

- The seat is heavy. Ask for help if required.

Folding down the steering column

#### Folding down the steering column

- Stand next to the mobility scooter.
- Squeeze the handlebar adjustment lever (T).
- Move the steering column as far as possible towards the seat.  
*Tip: when the seat is removed, the steering column can be folded down further.*
- Release the lever.

*Transport*

1. Turn the freewheel lever upwards.
2. Push the mobility scooter into the vehicle.
3. Switch off the mobility scooter.
4. Please ensure that the mobility scooter is properly secured.

**Storage when not using the mobility scooter**

**Attention:**


- Always store batteries fully charged.
- Fully charged gel batteries and AGM batteries will retain their charge for up to six months, but they must always be charged before use.
- Avoid storage of batteries at extremely high or low temperatures.
- Sealed gel batteries and AGM batteries do not need to be topped up with water. Contact a qualified scooter mechanic if you have any performance issues.

Longer than a week:

- Connect the battery charger at least once a week to maintain the condition of the batteries or put the mobility scooter away for the night with the battery charger connected.

Very long term (e.g. during winter or due to illness):

- Have your dealer disconnect the wiring of the batteries.

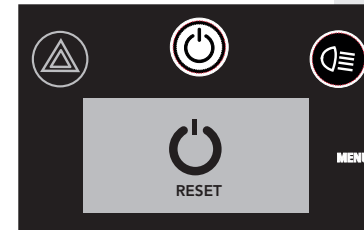Resetting

**General**

**Attention:**

- Never attempt to resolve a fault yourself. Contact your dealer instead.

1. If a fault occurs: the display will show up the fault.
2. Press the menu button to read the fault description.
3. Press the stand-by button to reset the controls: in many cases you will then be able to continue driving.
4. Contact your dealer to repair the error if resetting does not help.
5. Pass the error message on to your dealer.

## Warranty

### **Terms and Conditions of Warranty**

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product. Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated orally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

## Disposal of product and packaging

### **Disposal of product and packaging**

All parts of your mobility scooter are recyclable. Your dealer will know the collection points for disposal of the mobility scooter.

From an environmental perspective, we recommend that you dispose of the packaging and the mobility scooter by separating the waste as much as possible, allowing the waste materials to be recycled efficiently.

## Préface

*This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication. Life & Mobility does not accept liability for any errors in the text or the consequences thereof.*

*The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.*

*The information in this manual may not be reproduced and/or made public by any means without prior written permission from Life & Mobility.*

*Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.*

*No rights can be derived from this text. Subject to changes.*

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*© Life & Mobility – All rights reserved*