PRIMO

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Preface

Dear user,

In selecting a Life & Mobility Primo mobility scooter you have chosen for comfort, security and quality. The patented INDEGO suspension system guarantees unequalled driving pleasure. We hope you will drive many pleasant and safe miles. Embrace the freedom offered by Primo.

In this user manual you will find everything about the Primo, from safety instructions and operating functions to maintenance tips. Carefully read the user manual, so you can get the most out of your mobility scooter. Make sure that you do not lose the user manual for a possible future user.

We would like to keep you informed about updates, user experiences and other useful information. You can regularly read about the latest developments on the www.life-mobility.com website.

Kind regards,
Life & Mobility Management and Staff

Contents

Safety instructions	6
Introduction	11
Technical specifications	12
Product description	16
Settings and adjustments	21
General use	23
The Arrivo package	29
Using the Arrivo package	30
Maintenance	38
Transport and storage	42
Guarantee	44
Disposal of product and packaging	45

Safety instructions

As the user of the mobility scooter, you are always responsible for complying with the locally applicable safety instructions and guidelines. It is important to first acquire driving experience under the supervision of an experienced instructor. Your supplier can advise you on this.

Life & Mobility accepts no liability for any loss or injury caused by negligence or by failure to comply with the safety instructions.

Attention

- If you see this symbol in the manual, it indicates a supplementary instruction. Always follow this instruction. Negligence may adversely affect the operation of the mobility scooter and may cause physical injury or damage to the mobility scooter or the environment.

General

Warning

- Never change the construction of the mobility scooter.
- Keep the mobility scooter away from open fire.

Be careful

- Use the basket only to transport objects.
- Do not use your mobility scooter to pull vehicles, carts, etc.
- Remember that there are protruding parts, both fixed and loose, in and on the mobility scooter. These can cause damage to the surroundings and the mobility scooter.

Attention

- This mobility scooter is suitable for transporting a single person.
- Only use original Life & Mobility parts and accessories.
- The mobility scooter has a fully programmable electronic control system. Changes to the programming must only be made by Life & Mobility.
- **Never cover or remove the safety instructions and stickers.**
- Always switch off the mobility scooter and take out the ignition key before leaving it. This prevents unwanted use and/or theft.

Personal requirements

Warning

- *If you are under the influence of substances that may affect your ability to drive, you may not drive the mobility scooter.*

Attention

- Your vision must be adequate to drive the mobility scooter safely in any situation that may occur during use.
- Please follow the relevant traffic rules that apply.
- Make sure that you can always be seen by other road users. As a user of a mobility scooter you are very vulnerable on the road.
- Avoid quiet routes. Make sure that help can be arranged in the event of an emergency.
- Switch off the mobility scooter if you are using your mobile telephone. Your mobility scooter may activate security gates in shops.

Assembling and adjustment

(see also the chapter 'Settings and adjustments')

Warning

- Before using the mobility scooter, check that all components of the mobility scooter are properly mounted and secured.

Attention

- *Your dealer can set up various driving features for you.*

Sitting in the mobility scooter

Warning

- *Make sure that your fingers and limbs do not get trapped. Always be very careful:*
 - *when folding away armrests;*
 - *if you do not have complete control of your arms or legs.*

Driving the mobility scooter

(see also the chapter 'Use')

Warning

- Drive carefully on roads that are slippery due to rain, black ice or snow.
- Make sure that no loose clothing can get caught. It can obstruct the wheels and cause the scooter to overturn.
- Watch out for overturning:
 - *if you have set up the seat with a large back angle;*
 - *when driving up a slope;*
 - *when the slide system is in maximum rear position.*
- *Do not transport anything on the handlebars.*
- *Do not let anyone stand on the rear of your mobility scooter.*

Slopes and other obstacles

Warning

- Avoid slopes with a gradient greater than 6°. Driving on slopes (forwards or in reverse) is always dangerous.
- You must never free wheel down a slope.
- Drive carefully along (pavement) edges. An uneven surface increases the risk of overturning.

Attention

- Your centre of gravity moves when you drive up or down a slope. Please beware.
- Turn around carefully on a steep slope as there is a greater risk of overturning.
- Always mount and leave a pavement in a forward, straight line. Accelerate very gently. It is better to mount the pavement using a curb cut.
- The maximum bump height you can drive over is 10 cm.
- Soft, slippery or uneven surfaces increase the risk of a dangerous situation.
- Avoid prolonged driving on inclines. This can cause excessive wear on the engine and electrical components.

Transporting the mobility scooter without a driver

(see also the chapter 'Transport and storage')

Extremely dangerous

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.

Warning

- Make sure that your fingers do not get trapped when folding down the mobility scooter.

Maximum gradient

a forward, straight line

Approaching a pavement

- Never carry the mobility scooter by the plastic parts of the housing.

Attention

- Make sure that the mobility scooter is fully assembled before use.

Miscellaneous

Warning

- Do not put the mobility scooter in direct sunlight: the upholstery can become too hot, and cause burns or irritation to the skin.

Be careful

- Do not allow the mobility scooter to come into contact with seawater: seawater is corrosive and will damage the mobility scooter.
- Do not allow the mobility scooter to come into contact with sand: sand may penetrate to the moving parts of the mobility scooter, causing faster wear and tear.

Attention

A number of technical precautions have been implemented to further guarantee your safety:

- the immobiliser is activated whenever a battery charger is connected;
- the speed in bends is limited automatically.
- the freewheel is limited 2,5 mph (4 km/h);
- the display warns you if the battery voltage is too low;
 - via the battery indicator in the standard model
 - via the display in the Arivo package
- the display gives you information to resolve possible faults;
- in the event of an electronic fault, the mobility scooter will make a controlled stop;
- the mobility scooter's control panel includes an emergency stop button to stop the scooter quickly in an emergency;

Introduction

- *the mobility scooter has ergonomically-shaped safety handlebars to prevent injuries to fingers and hands;*
- *to prevent parts of the body becoming jammed, the necessary components have been covered;*
- *to secure your body in the correct position, your supplier can fit a seatbelt to the fastening points of the armrests that are attached to the backrest.*

EMC-approved

The mobility scooter is approved for EMC (Electro-Magnetic Compatibility). Tests have shown that mobility scooters are not sensitive or damaging to mobile communication equipment and electronically controlled equipment. Despite this, please be careful when using the mobility scooter and mobile or electronic equipment at the same time.

The Primo is an electrically-powered mobility scooter, suitable for both young and older people who have difficulty walking long distances.

You can set up the Primo mobility scooter to meet all your requirements. This guarantees a relaxed posture while driving. The mobility scooter is basically operated with two hands. You need not be strong to drive the mobility scooter and the controls are easy. The Primo is available as a 3-wheel and 4-wheel scooter, and provided with a Arrivo package if necessary.

This manual describes the default settings of your mobility scooter. A list has been included in the back of this user manual for recording your specific settings.

Technical specifications - Primo 3

- Intended use: outdoors, between -25° C and +50° C, on paved paths and roads
- L x w x h: 125 x 66 x 105 cm
- L x w x h (without seat): 125 x 66 x 88 cm
- Weight without batteries (with 60 Ah battery): 75 kg (110 kg)
- Weight of heaviest to dismantle component (seat): 15 kg
- Maximum static stability down slope, up slope and sideways up slope: 9 °
- Maximum dynamic stability up slope: 6 °
- Maximum safe slope angle: 6 °
- Max. driving distance with fully-charged batteries*: 25 miles (30 km) (50 Ah)/31 miles (40 km) (60 Ah)
- Maximum obstacle height: 10 cm
- Maximum obstacle to drive off (pavement): 10 cm
- Max. (programmable) speed: HI = 7,5 mph (12 km/h), LO = 4 mph (6 km/h). Less is possible.
- Seat depth: 44 cm
- Seat width: 48 cm
- Seat height above footrest (min. - max.): 42.5 - 45 - 47.5 - 50 cm
- Height of footrest above ground: 17 cm
- Turning circle (radius): 120 cm
- Max. user weight: 160 kg
- Battery capacity (optional): 50/60** Ah
- Max. load: 8 amps
- Motor, transaxle: 500 watts 24 V
- Magnetic brake: 8 Nm 24 V
- Wheel size: 4.10/3.50 – 6"
- Complies with: EN 12184 Class B

Lights

- Headlight, rear light, brake lights, indicator: 24V/5W

The mobility scooter has 2 12-volt dry (MK) batteries. The following types are suitable:

- 50 Ah (198 x 166 x 171 mm) AGM
- 60 Ah (259 x 169 x 178 mm) Gel

* Depending on the set speed, outside temperature, weight of the user, tyre pressure, etc.

** Gel batteries

If a different battery is used than supplied, the following dimensions are the maximum external measurements:

- 278 x 175 x 190 mm
- 261 x 135 x 230 mm
- 238 x 140 x 235 mm

Test specifications

- The scooter meets the following standards: ISO 7176- 8,9,14,16
- Requirements and testing methods for static, shock and fatigue strength: ISO 7176-8
- Climate test for scooters: ISO 7176-9
- Propulsion and steering systems for scooters: ISO 7176-14
- Requirements for resistance to ignition: ISO 7176-16

Service and technical support

Contact your supplier if you have any questions regarding specific settings, maintenance or repairs. Always have the serial number ready. You can find these on the mobility scooter's CE sticker (see also page 20).

Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

Technical specifications - Primo 4

- Intended use: outdoors, between -25° C and +50° C, on paved paths and roads
- L x w x h: 125 x 66 x 105 cm
- L x w x h (without seat): 125 x 66 x 88 cm
- Weight without batteries (with 60 Ah battery): 85 kg (120 kg)
- Weight of heaviest to dismantle component (seat): 15 kg
- Maximum static stability down slope, up slope and sideways up slope: 9 °
- Maximum dynamic stability up slope: 6 °
- Maximum safe slope angle: 6 °
- Max. driving distance with fully-charged batteries*: 25 miles (28 km) (50 Ah)/31 miles (38 km) (60 Ah)
- Maximum obstacle height: 10 cm
- Maximum obstacle to drive off (pavement): 10 cm
- Max. (programmable) speed: HI = 7,5 mph (12 km/h), LO = 4 mph (6 km/h). Less is possible.
- Seat depth: 44 cm
- Seat width: 48 cm
- Seat height above footrest (min. - max.): 42.5 - 45 - 47.5 - 50 cm
- Height of footrest above ground: 17 cm
- Turning circle (radius): 165 cm
- Max. user weight: 160 kg
- Battery capacity (optional): 50/60** Ah
- Max. load: 8 amps
- Motor, transaxle: 500 watts 24 V
- Magnetic brake: 8 Nm 24 V
- Wheel size: 4.10/3.50 – 6"
- Complies with: EN 12184 Class B

Lights

- Headlight, rear light, brake lights, indicator: 24V/5 W

The mobility scooter has 2 12-volt dry (MK) batteries. The following types are suitable:

- 50 Ah (198 x 166 x 171 mm) AGM
- 60 Ah (259 x 169 x 178 mm) Gel

* Depending on the set speed, outside temperature, weight of the user, tyre pressure, etc.

** Gel batteries

If a different battery is used than supplied, the following dimensions are the maximum external measurements:

- 278 x 175 x 190 mm
- 261 x 135 x 230 mm
- 238 x 140 x 235 mm

Test specifications

- The scooter meets the following standards: ISO 7176- 8,9,14,16
- Requirements and testing methods for static, shock and fatigue strength: ISO 7176-8
- Climate test for scooters: ISO 7176-9
- Propulsion and steering systems for scooters: ISO 7176-14
- Requirements for resistance to ignition: ISO 7176-16

Service and technical support

Contact your supplier if you have any questions regarding specific settings, maintenance or repairs. Always have the serial number ready. You can find these on the mobility scooter's CE sticker (see also page 20).

Expected service life

Life & Mobility assigns a service life of 6 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

Product description

Checking the delivery


Before using the mobility scooter, check if the delivery matches the product description below.

If you discover any transportation damage upon delivery, please inform your supplier. Life & Mobility trusts that the mobility scooter meets your expectations and hopes that it brings you much convenience.


Product description

Mobility scooter (Primo 3 and Primo 4)


- A. Handlebars with control panel
- B. Basket
- C. Headlight
- D. Indicators (orange)/ Brake lights / Rear light (red)
- E1. Front wheel
- E2. Rear wheel
- F. Footrest
- G. Battery cover
- H. Freewheel lever (limited)
- I. Anti tip support (exclusive)
- J. Folding armrests
- K. Seat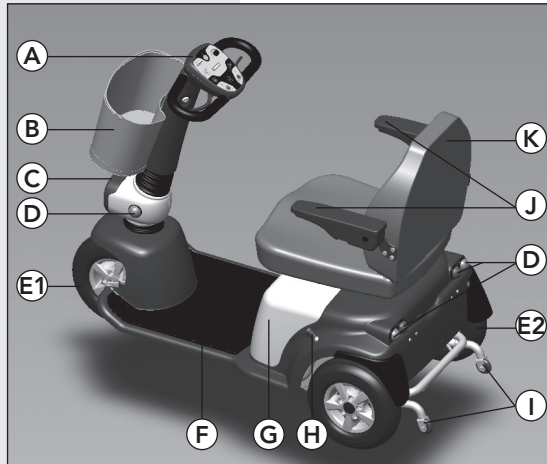


Product Description - Primo 3

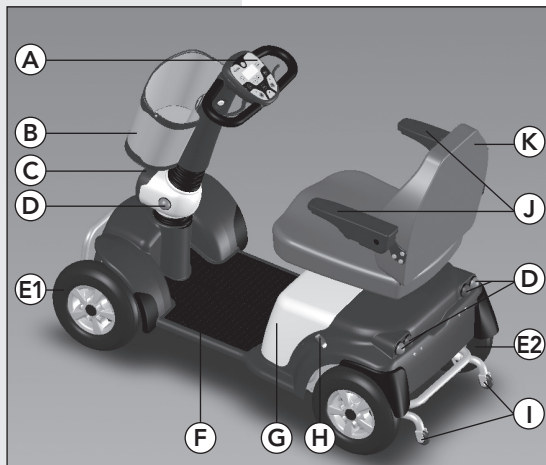
Product Description - Primo 4

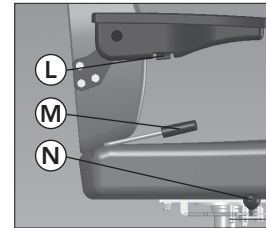
Product description - seat

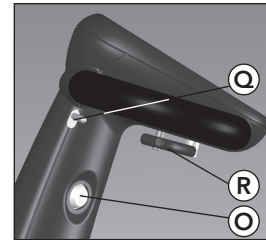Product description handlebars

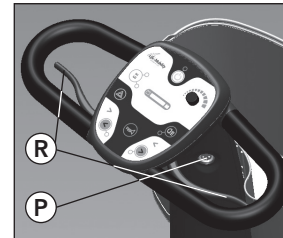Product description handlebars

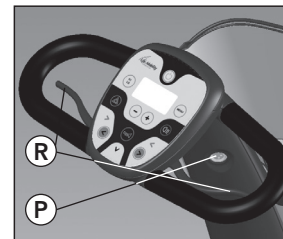Product description handlebars with Arrivo package

Seat

- L. Armrest adjusting wheel
- M. Back angle lever
- N. Seat lever (left or right)* (rotate)

Handlebars


- O. Ignition lock (left or right)*
- P. Battery charger contact (left or right)*
- Q. Handlebar angle adjustment lever (left or right)*
- R. Throttle lever/Reverse lever (left or right)*

* Depending on how your mobility scooter has been set up.

Standard control panel


- S. Stand-by button/Stop button
- T. HI/LO function
- U. Warning lights
- V. Indicator right
- W. Indicator left (with indicator light)
- X. Horn
- W. Indicator right (with indicator light)
- Z. Indicator left
- a. Lighting
- b. Battery indicator/error codes
- c. Speed level
- d. Indicator light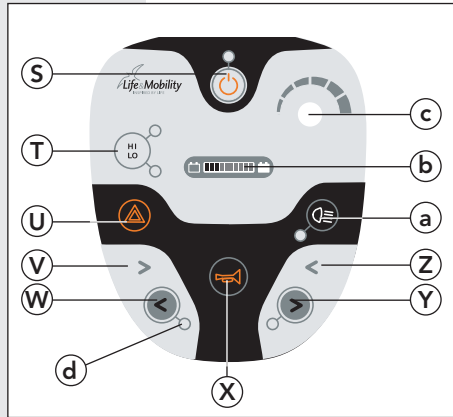


Standard control panel

Control panel Arrivo package

- S. Stand-by button/Stop button (also see page 34)
- T. Display (Readable in display: mph (km/h), trip distance, total miles (km), date and time)
- U. HI/LO function
- V. Reduce speed button
- W. Increase speed button
- X. Warning lights
- Y. Indicator right
- Z. Indicator left
- a. Reverse button
- b. Indicator right
- c. Indicator left
- d. Horn
- e. Lighting
- f. Menu button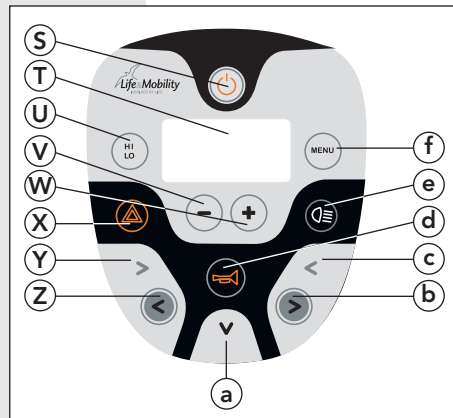

Control panel Arrivo package

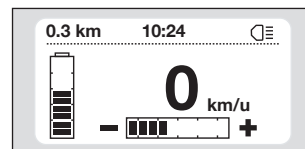Display

Features

- Torsion springs on front and rear wheels
- INDEGO suspension system on rear axle (and front axle in four-wheeler)
- Comfortable seat with pre-shaped seating surface and backrest
- Easy and light operation of handlebars and control levers
- Clear battery display and low battery voltage warning
- Provided with stand-by button to (de) activate driving status

Safety and comfort facilities

- Ergonomically-shaped handlebars with soft lining to adjust angle via a gas spring
- Automatic speed limiter in bends
- Indicator turns off automatically after bend (Arrivo package)

Not pictured

Various accessories are optionally available, including:

- Captain Seat (Arrivo package);
- padded armrests;
- slide system under chair;
- seatbelt;
- additional foot pedal accelerator (only possible in combination with Arrivo package);
- rear-view mirror;
- eye anchor fixing;
- crutch holders;
- rollator frame holder;
- lined, waterproof and wind-resistant lap blanket;
- waterproof cover.

See www.life-mobility.com for more accessories.

Settings and adjustments


Markings on the mobility scooter

There are a number of stickers on the mobility scooter. Below is an overview of these stickers, with their location.

CE sticker

The Primo complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices. and has a CE label.

The CE sticker can be found under the battery cover (G).

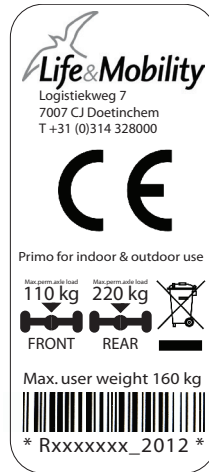
CE sticker

Attention

- Always replace the battery cover as this protects the electronics.
- To ensure that the cover is tightly closed, first attach the top of the battery cover and then the bottom.

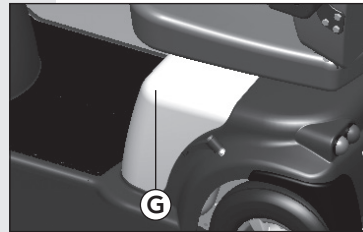Freewheel sticker (see also page 21)

The sticker above the freewheel lever indicates the "push position". In this setting the mobility scooter can be moved freely.

The sticker below the free-wheel lever indicates that the mobility scooter is ready to be electrically driven.

Battery connection diagram


The battery connection diagram is located on the inside of the battery cover. It also includes the fuse and the motor controls.

Push position

Electric driving position

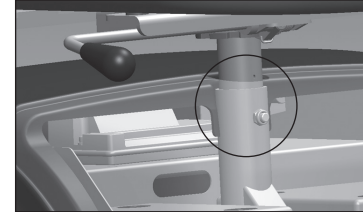
Adjusting seat height
(battery cover removed)

Back angle

Attention

- Only adjust the backrest when you are sitting in the seat.
- The maximum back angle (180°) is only intended for transporting the seat.
- Watch out for overturning:
 - if you have set up the seat with a large back angle;
 - when driving up a slope;
 - when the slide system is in maximum rear position.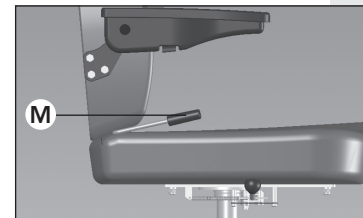

Change back angle

- Sit down on the seat.
- Lift the lever (M).
- Use your upper body to push the backrest backwards into the desired position, or:
- Bend your upper body forwards to allow the backrest to move forward automatically.
- Release the lever (M).
The backrest is now locked in its current position.


Armrests

Attention

- The position of the armrests changes if the position of the backrest is adjusted.

1. Rotate the armrest upwards to the desired angle.
2. Use the adjusting wheel (L) to lock the angle of the armrest in the correct position.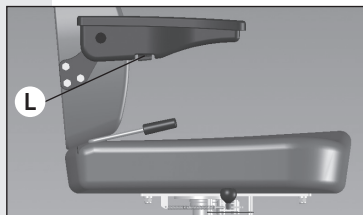

Change armrest angle


Adjusting the handlebars

Attention

- Do not adjust the angle of the steering column while driving. A gas spring allows the handlebars to move forward independently.

Steering column angle

1. Sit down on the seat.
2. Squeeze the lever (Q)*.
3. Pull the steering column towards you or push it away until you reach the desired position.
4. Release the lever.
The steering column is now locked in place.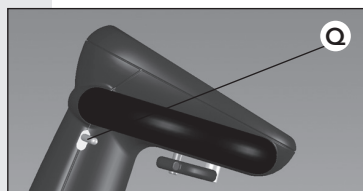

Change handlebar angle

* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.

Check the following before driving off:

- whether the lights are working;
- the battery status: battery level indication in display (see page 24 and 36);
- the tyre pressure (see chapter on 'Maintenance');
- whether the freewheel lever is pointing down (electric driving position, see below);
- whether the battery charger has been removed.

Getting in and out by yourself


Attention

- Make sure that the seat is properly locked in place before driving off.

Getting in/out

1. Push the seat lever (N)* down.
2. Rotate the seat a quarter turn to the left or to the right.
3. Get in or out.
4. Rotate the seat back again until you hear a click.

The seat is now locked in place.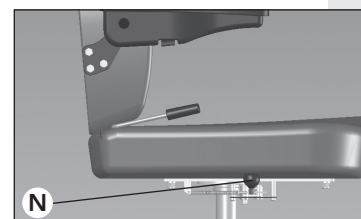


Turn seat

Freewheel lever

Warning

- Never use this function on a slope or while driving. It will disable the brakes. Always turn the freewheel lever downwards to re-engage the brakes after using the freewheel function.

1. Turn the freewheel lever (H) upwards to move the mobility scooter by hand.
2. Turn the freewheel lever down for the electric drive position. Teach yourself to do this each time you have used it.

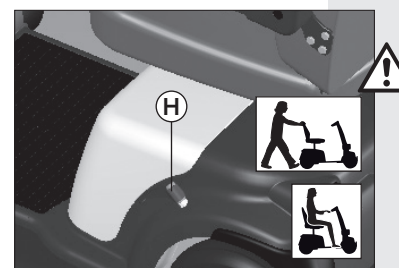Freewheel lever

* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.

Starting

1. Insert the key in the ignition lock (O)*.
2. Turn the key a quarter turn clockwise.
3. Wait until the LED bar is lit.

The mobility scooter is now ready to drive.

Speed

The speed can be adjusted for either indoor or outdoor use.

Warning

- Adjust your speed before steering into a bend. The bend speed limiter will not adjust your speed until the bend is too tight. The scooter may overturn.
- Despite the small turning circle of the mobility scooter, it may still be difficult to turn into lifts or doorways. Make sure that you always use the correct low speed setting for tight manoeuvring.



Attention

- Adjust your speed using the throttle lever. The speed settings determine how fast you can go if you squeeze the throttle lever all the way.
- Check that the road behind you is clear when reversing.
- The mobility scooter reverses more slowly than when it drives forward.
- An alarm can be heard when reversing.
- On request, your supplier can change the throttle lever control (method).

The maximum speed is (HI) 7,5 mph (12 km/h). Your supplier can adjust the speed setting downwards.

Ignition lock

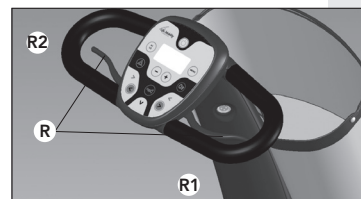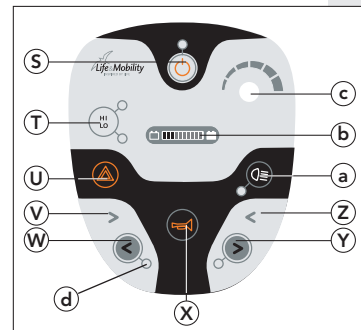** This is located on the left or right-hand side, depending on the configuration of your mobility scooter.*

Turn button (c) to adjust the maximum speed level.

Choose a suitable speed level for each situation.

Driving forward and reversing

1. Slowly squeeze the throttle lever (R1)*.
The mobility scooter starts moving forwards.
2. Squeeze the throttle lever further.
The mobility scooter will start moving forward more quickly.
3. Release the throttle lever.
The mobility scooter will stop. The brake lights are activated.
4. Use throttle lever (R2)* to reverse.

Stand-by button/stop button

The stand-by button/stop button (S) has two functions:

- The mobility scooter is temporarily 'switched off'. You can then reach over the handlebars safely without turning the ignition key. The drive controls are switched off, preventing you from driving off unintentionally.
- If you use the stand-by button/stop button while driving, you will come to a standstill more quickly.

Lights

1. Press the (a) button to switch the lights on or off.
2. Press the (W) or (Y) button to switch the indicators on or off.
3. Press the (U) button to switch the warning lights on or off.

** This is located on the left or right-hand side, depending on the configuration of your mobility scooter.*



Batteries

Reading the battery status

The battery-shaped indicator on the display shows the remaining battery capacity:

Battery status	Indicator
All red and green blocks are visible.	The batteries are full.
3 red blocks are visible	You can drive another 3 - 6 miles (5 - 10 km) (depending on the type of battery and other circumstances). You should start charging now.
1 block visible	Recharge the mobility scooter as soon as possible.

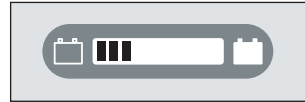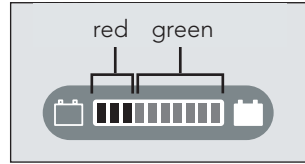Charging batteries


1. Insert the battery charger plug into the battery charger contact (P)*.
2. Insert the mains cord plug in a wall socket.
3. Always fully complete the charging cycle of 8 - 10 hours.

Be careful

- New batteries must be run in. Batteries only reach full capacity after they have been used and charged a few times. Take this into account during your first few trips.
- Charge the batteries daily if you drive a lot with your mobility scooter and once every few days if you don't drive often. This extends the service life.
- Always charge the batteries to full capacity.
- Avoid draining the batteries completely. The mobility scooter will lose power which will have a negative effect on the service life of the battery.

* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.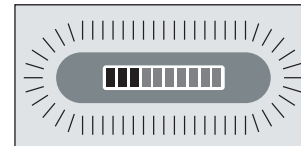

- Recharge fully drained batteries as soon as possible.
- Do not keep your batteries fully drained for a prolonged period of time.
- Never put metal parts on the batteries: this may short-circuit the batteries and damage them.

Attention

- At lower temperatures the capacity of the batteries is reduced. This also reduces the operating range.
- Always connect the charger when putting the mobility scooter away for the night.
- You cannot overcharge the batteries supplied. Thanks to a voltage limiter, the charger automatically switches off when the batteries are full.
- Only use Life & Mobility battery chargers.
- If you use batteries other than the batteries supplied by Life & Mobility ensure that a good retaining ring is fitted to prevent them moving. If a retaining ring is not fitted this could lead to short-circuiting.

Parking

When parking your scooter, place a chain lock through one of the wheels for added security.

Error codes

When there is a error message, the LED Bar will flash.

Error message	Indicator
1 block flashes ■	The battery is empty or connected incorrectly. Check whether the battery is connected correctly. Then try to charge the battery again.

The Arrivo package

2 blocks flash ■■	The electrical connection to the engine has been interrupted. Have your supplier check all engine electrical contacts.
3 blocks flash ■■■	Short circuit between engine and battery. Contact your supplier.
4 blocks flash ■■■■	The freewheel is engaged or the brake has been disengaged manually. Check the position of the freewheel lever.
5 blocks flash ■■■■■	Not in use.
6 blocks flash ■■■■■■	The brake has intervened. For example because the charger is still connected or the seat is not in the correct driving position.
7 blocks flash ■■■■■■■	Error in operating the throttle lever. For example because this was not in the 0 position during starting.
8 blocks flash ■■■■■■■■	Electronics error. Contact your supplier.
9 blocks flash ■■■■■■■■■	The parking brake has a faulty electronic contact. Contact your supplier.
10 blocks flash ■■■■■■■■■■	Too high voltage. Contact your supplier.

The Arrivo package comprises:

Captain seat provided with slide system, high back support, sideways support, adjustable lumbar support, black imitation leather upholstery and headrest mounting.

Arrivo control panel with illuminated display, digital speedometer & and battery indicator meter, total miles/km & trip distance, time & date

Various adjustments regarding driving features, throttle handling, bend sensor, indicators that switch off automatically, reversing signal.

Use of the Arrivo package

You can adjust the **Captain seat** entirely as desired. It is advisable to contact your dealer for the correct settings.

Adjust back angle


Attention:

- Only adjust the backrest when you are sitting in the seat.
- The maximum back angle (180°) is only intended for transporting the seat.
- Watch out for overturning:
 - If you have set up the seat with a large back angle.
 - When driving up a slope.
 - When the slide system is in maximum rear position.


1. Sit down on the seat.
2. Lift the lever (O).
3. Use your upper body to push the backrest backwards into the desired position, or:
4. Bend your upper body forwards to allow the backrest to move forward automatically.
5. Release the lever (O).
The backrest is now locked in its current position.

Change back angle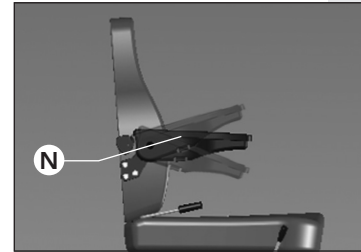

Change armrest angle

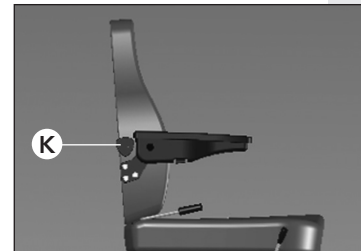Set backrest lumbar support

Armrests

Attention:

- The position of the armrests changes if the position of the backrest is adjusted.

1. Rotate the armrest upwards to the desired angle.
2. Rotate the adjusting wheel (N) to lock the angle of the armrest.
3. Loosen the adjusting wheel (N) to move the armrest down.

Backrest lumbar support

1. Turn the button (K) once at a time. This changes the curve of the backrest.
2. Check whether the new setting has the desired result.
There are 3 positions.

Starting

1. Insert the key in the ignition lock (O)*.
2. Turn the key a quarter turn clockwise.
3. Wait until '0 mph' (0 km/h) appears on the display.


The mobility scooter is now ready to drive.

Ignition lock

Speed

Use the throttle lever to control the speed. With the HI/LO function you can change directly from high speed (max. 7,5 mph, 12 km/h) to low speed (max. 4 mph, 6 km/h) and vice versa.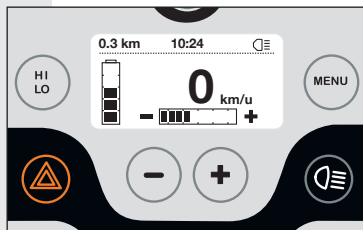

Ready for use


Warning

- Adjust your speed before steering into a bend. The bend speed limiter will not adjust your speed until the bend is too tight. The scooter may overturn.
- Despite the small turning circle of the mobility scooter, it may still be difficult to turn into lifts or doorways. Make sure that you always use the correct low speed setting for tight manoeuvring.

Attention

- Adjust your speed using the throttle lever. The speed settings determine how fast you can go if you squeeze the throttle lever all the way.
- Check that the road behind you is clear when reversing.
- The mobility scooter reverses more slowly than when it drives forward.
- An alarm can be heard when reversing.
- On request, your supplier can change the throttle lever control (method).

* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.

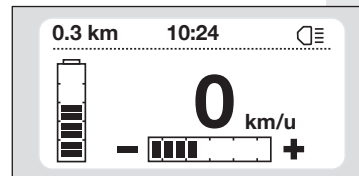Default view speedometer

Indoors, in shops, etc.

On the pavement

In busy streets

On quiet streets

By default the speedometer is set to 4 blocks, which is 40% of the total power. The maximum speed is 7,5 mph (12 km/h). Your supplier can adjust the speed setting.

1. Press the + or - button to adjust the speed setting.

Choose a suitable speed setting for each situation.

- Indoors, in shops, etc., 1 to 2 blocks
- On the pavement: up to 4 blocks
- In busy streets: up to 7 blocks
- On quiet streets: up to 10 blocks

Driving forward and reversing

1. Slowly squeeze the throttle lever (R1)*.
The mobility scooter starts moving forwards.
2. Squeeze the throttle lever further.
The mobility scooter will start moving forward more quickly.
3. Release the throttle lever.
The mobility scooter will stop. The brake lights are activated.
4. Use throttle lever (a)* to reverse.


* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.

Stand-by button/stop button

The stand-by button/stop button (S) has two functions:

- The mobility scooter is temporarily 'switched off'. You can then reach over the handlebars safely without turning the ignition key. The drive controls are switched off, preventing you from driving off unintentionally.
- If you use the stand-by button/stop button while driving, you will come to a standstill more quickly.

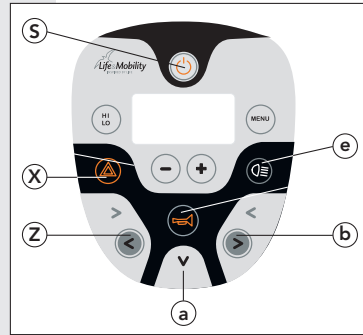Lights

Attention

- After a tight bend the indicators automatically switch off.
- Your supplier can set up the one-handed control for you.
- Your supplier can disable the default sound from the warning lights.

1. Press the (e) button to switch the lights on or off.
2. Press the (Z) or (b) button to switch the indicators on or off.
3. Press the (X) button to switch the warning lights on or off.

User menu


1. Start screen

By default the (home) screen of the control panel displays the following information:


- the distance covered (A);
- the time (B);
- lights on/off (C);
- the battery status (D);
- the speed (E).


Stand-by button/stop button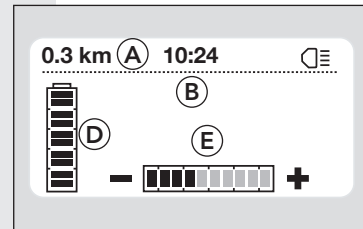


Display HI

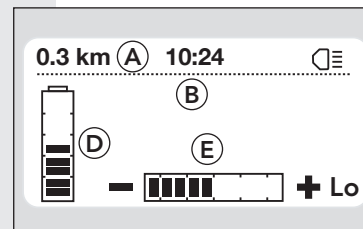Display LO

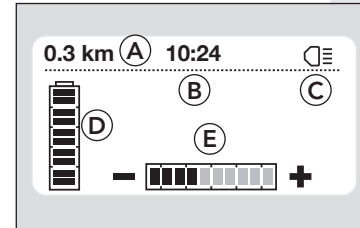Home screen

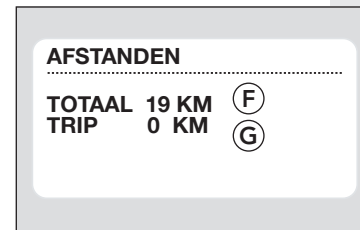Distances menu

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Setting the time & date

2. Press the menu button.

The second menu (distances) is displayed and shows the following:

- the total distance covered (F);
- the journey distance (G).

3. Press the menu button again.

The third menu is displayed and shows the following:

- the time and date (H).

4. Wait a few seconds.

The screen automatically jumps back to the home screen.

Setting the journey counter to 0

1. In the second menu (distances) hold the menu button down for 2 seconds.

The journey counter will be reset to 0.

Setting the time and date

1. Select the third menu (time & date).
2. Then press and hold the menu button.
The 'hours' will start to flash in the edit menu.
3. Use the + and - buttons to set the correct hour.
4. Press the menu button again to save the hours.
The minutes will start flashing.
5. Use the + and - buttons to set the correct minutes and press the menu button.
The date will start flashing.
6. Use the + and - buttons to set the correct day, month and year.
7. Press the menu button to confirm.
You will return automatically to the third menu (time & date).


Reading the battery status


The battery-shaped indicator on the display shows the remaining battery capacity:

Battery status	Indicator
All blocks visible	The batteries are full.
3 blocks visible	You can drive another 3 - 6 miles (5 - 10 km) (depending on the type of battery and other circumstances). You should start charging now.
1 block visible	Recharge the mobility scooter as soon as possible. The screen gives a warning.


See page 26 and page 27 for more information about batteries.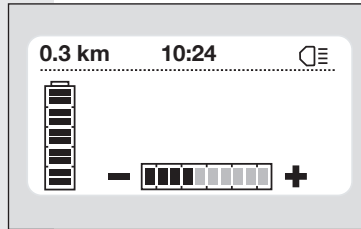

Battery status: full

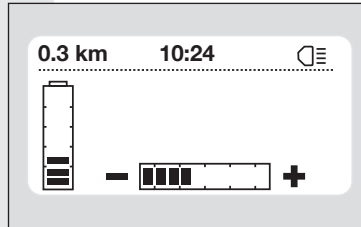
Battery status: 3 - 6 miles remaining (5 - 10 km)

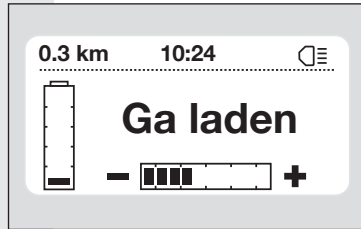Battery status: empty

Battery charger contact

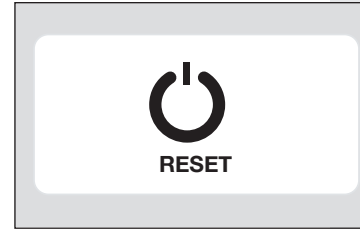Resetting

Faults

Attention

- Never attempt to resolve a fault yourself. Contact your supplier.

- If a fault occurs: the display will show up the fault.
- Press the menu button to read the fault description.
- Press the stand-by button to reset the controls: in many cases you will then be able to continue driving.
- Contact your dealer to repair the fault if resetting does not help.
- Pass on the error message to your dealer.

Maintenance

Cleaning

Be careful

- Do not use bleach or solvents.
- Do not use a high-pressure cleaner to clean the mobility scooter.

Attention

- Clean the upholstery and the body shell regularly.

Upholstery

1. Remove any light dirt with a solution of a mild cleaning agent and warm water.
2. Carefully scrub the upholstery with a soft brush if the dirt has been absorbed into the upholstery.

Body shell

1. Clean the housing with a non-abrasive cleaning product.
2. Apply a non-abrasive car wax to the housing.

Batteries (see also the chapter 'Use')

Be careful

- *Batteries can be dangerous. Only allow your batteries to be installed and maintained by a qualified scooter mechanic. They have the required qualifications and tools to perform the installation and maintenance safely and correctly.*
- The scooter uses two batteries with a 24-volt series connection. Never use both batteries at the same time:
 - *if they are from different manufacturers;*
 - *if they use different technologies;*
 - *if they have different date codes;*
 - *gel in combination with AGM batteries;*

Attention

- Always fully charge batteries before using them for the first time.
- Fully recharge the batteries at least once a month.
- The battery connection method is illustrated on a diagram on the inside of the battery cover.
- Be careful with damaged batteries. Leave the (environmentally-friendly) replacement of batteries to the supplier.

Tyres

Warning

- For safety reasons always ensure that the tyre pressure on your mobility scooter is correct. If the tyre pressure is too low, road holding will deteriorate, the operating range will be reduced and the steering will become heavier.

Attention

- Contact the supplier in the event of a flat tyre.
- The mobility scooter tyre pressure must be sufficient. Check this regularly.
- If the tyre pressure is too low you will not be able to drive as far.
- The pressure of the front tyre must be at least 2.0 bar.
- The pressure of the rear tyres must be at least 2.5 bar.
- The maximum tyre pressure is 3.5 bar.
- If the tyre pressure is too high, driving will become uncomfortable.
- Regularly check the tyres for wear and tear (due to drying out).

Service and technical support

Have the mobility scooter checked and serviced by your supplier at least once a year. Preventive maintenance ensures that the proper and correct functioning of all moving parts is checked, as well as the electronics and the condition of the batteries.

Repairs should only be performed by your dealer or by Life & Mobility, and only original parts supplied by Life & Mobility should be used for this.

Maintenance matrix

The following matrix indicates which parts should be regularly checked. If a defect occurs, contact your dealer to resolve this issue.

Part	Action	Daily	Weekly	Monthly	Each quarter	Annually
Battery charger	Check functioning	X				
Tyres	Check tyre pressure (front: 2.0 bar, rear: 2.5 bar)		X			
Lights	Check functioning	X				
Housing	Clean with damp cloth		X			
Wiring	Check for breakages		X			
Moving parts	Check functioning and play			X		
Batteries	Full charging cycle			X		
Parking brake	Check functioning				X	
Tyres	Check wear and tear				X	
Mobility scooter	Servicing by your dealer					X

Transport and storage

Transporting the mobility scooter

Extremely dangerous

- When transporting the mobility scooter in a motor vehicle, always make sure you are seated in a fixed seat of the motor vehicle. Mobility scooters may never be used as seating support in a motor vehicle, even if the mobility scooter is secured with a taxi restraint system.


Be careful

- Never carry the mobility scooter by the plastic parts of the housing.

Folding down the backrest

- Stand next to the mobility scooter.
- Grip the backrest firmly.
- Lift the lever (M).
- Push the backrest forward.
- Release the lever (M).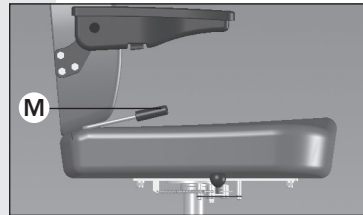

Folding down the backrest

Removing the seat

- Remove the tenon from the mortise under the seat.
- Turn the seat a little.
- Lift the seat from the mobility scooter.


Attention

- The seat is heavy. Ask for help if required.


Folding down the steering column

- Stand next to the mobility scooter.
- Squeeze the handlebar adjustment lever (Q)*.
- Move the steering column as far as possible towards the seat.
Tip: if the seat is removed, the steering column can be folded down further.
- Release the lever.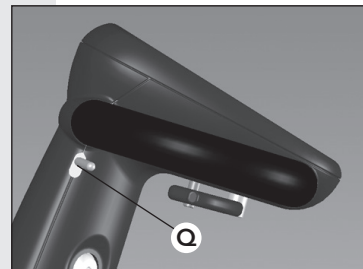


Folding down the steering column


* This is located on the left or right-hand side, depending on the configuration of your mobility scooter.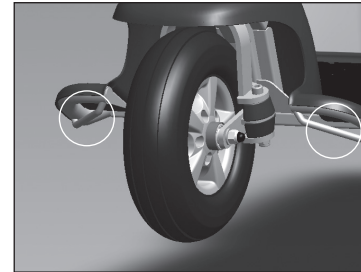

Primo 3
Position front eye anchor fixing

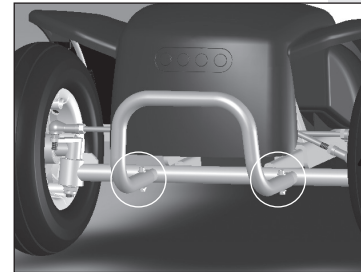Primo 4
Position front eye anchor fixing

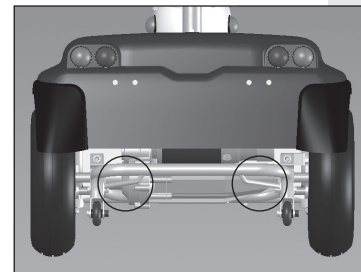Position back eye anchor fixing

Transport

- Turn the freewheel lever upwards.
- Drive the mobility scooter into the vehicle.
- Switch off the mobility scooter.
- Please ensure that the mobility scooter is properly secured.

The images indicate the location of the eye anchor fixing for securing the mobility scooter.

Storage when not using the mobility scooter

Attention

- Always store batteries fully charged.
- Fully charged gel batteries and AGM batteries will retain their charge for up to six months, but they must always be charged before use.
- Avoid storage of batteries at extremely high or low temperatures.
- Sealed gel batteries and AGM batteries do not need to be topped up with water. Contact a qualified scooter mechanic if you have any performance issues.

Longer than a week

- Connect the battery charger at least once a week to maintain the condition of the batteries or put the mobility scooter away for the night with the battery charger connected.

Very long term (e.g. during winter or due to illness)

- Have your supplier disconnect the battery wiring.

Warranty

Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product. Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated orally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

Disposal of product and packaging

Disposal of product and packaging

All parts of your mobility scooter are recyclable. Your supplier will know the collection points for disposal of the mobility scooter.

From an environmental perspective, we recommend that you dispose of the packaging and the mobility scooter by separating the waste as much as possible, allowing the waste materials to be recycled efficiently.

This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication. Life & Mobility does not accept liability for any errors in the text or the consequences thereof.

The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.

The information in this manual may not be reproduced and/or made public by any means without prior written permission from Life & Mobility.

Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.

No rights can be derived from this text. Subject to changes.

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© Life & Mobility – All rights reserved