










Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
<b>MORGAN</b> VFS 	Front	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	
	Drive	N/A	M10 Nylon Nut	47 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
		680 (green)	Motor & Hub	NA	
	Rear	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	



LEON<sup>VRS</sup>

Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
LEON <sup>VRS</sup>	Front	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	
	Drive	N/A	M10 Nylon Nut	47 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
		680 (green)	Motor & Hub	NA	





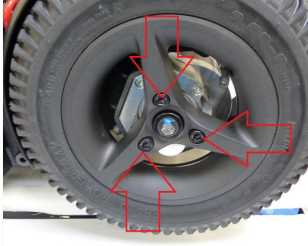

Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
 <p><b>KAMELEON</b></p>	Back	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	
	Drive	N/A	M10 Nylon Nut	47 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
		680 (green)	Motor & Hub	NA	





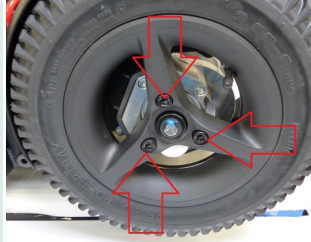



Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
<b>MID</b> <b>Lectus</b>  	Front	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	
	Drive	2701(green)	Hex bolt M8*20L	24 Nm	
		243	3x M10x16L	24 Nm	
	Rear	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	



Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
<b>EVO</b> <b>Lectus</b>  	Drive	2701(green)	HEX Bolt M8*20L	24 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
	Rear	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	



Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
<b>EVO</b> <b>Altus</b>  	Drive	2701(green)	HEX Bolt M8*20L	24 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
	Rear	243 (Blue)	Countersunk Screw M6*20L	12.7 Nm	








Type chair

BLAZER





Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
BLAZER	Front	NA	Hex Bolt / M10*100L + Nylon Nut	NA	
	Drive	N/A	M10 Nylon Nut	47 Nm	
		243 (Blue)	3x M10x16L	24 Nm	
		680 (green)	Motor & Hub	NA	

Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
 <p>ERGO Stand</p>	Drive	N/A	M10 Nylon Nut	47 Nm	
		N/A	Hex Bolt M8*25	NA	
	Rear	N/A	(Button head socket screw) M6*20L"	NA	





Type chair	Wheels	Loctite No.	Type bolt/screw	Torque (Nm)	
 <p><b>ERGO Traveller</b></p>	Drive	N/A	M10 Nylon Nut	47 Nm	
	Front	N/A	Hex Bolt /M8*85 + Nylon Cap Nut	NA	