CANTO<sup>next</sup>

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Foreword	4
Product description	6
Technical specifications	8
Optional	10
Safety instructions	12
Settings and adjustments	15
Making the wheelchair smaller	23
Use	24
Transporting the Canto Nxt	27
Maintenance	31
Technical problems	32
Guarantee	34

## Foreword

The Canto Nxt is a tilting wheelchair that has been specially designed for permanent wheelchair users who need a wheelchair with maximum comfort. The Canto Nxt always provides the care professional with a customized personal solution, irrespective of the weight, body dimensions and state of health of the passive user.

The Canto Nxt has two gas springs which give the wheelchair a stable, comfortable tilting frame which, in addition to a large tilting range, is light to operate with or without a user. The backrest angle is adjustable. The arm and legrests are easy to remove when transferring to and from the chair. The Canto Nxt can be made smaller.

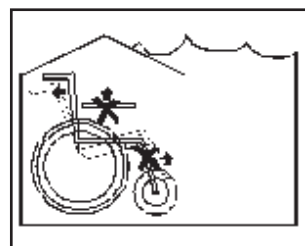
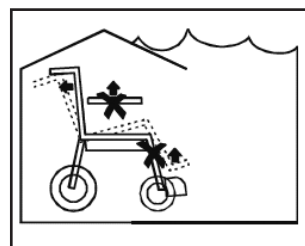
The Canto Nxt is constructed according to a modular system so that the chair can be put together on the basis of the sitting comfort, dimensions, variant and drive requirements of the user and care professional. The basis of this modular system is a multifunctional basic frame which can be fitted with different wheel combinations, seats and backrests.

The Canto Nxt is water splash resistant and suitable for both indoor and outdoor use.

Depending on the model, the maximum user weight is 150 kg\*. This is indicated on the CE sticker.

The Canto Nxt has been tested positively in compliance with the relevant requirements of the European wheelchair standards EN 12182 and EN 12183 and as a 'crash test safe wheelchair' in accordance with ISO 7176-19:2008 (see page 27).

\* In combination with a Kelvin seat, the maximum user weight is 125 kg

The Canto Nxt therefore complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices.

**Important:** please read these instructions for use carefully before using the Canto Nxt for the first time.

**Important:** the optional extras for the Kelvin seat element are described in the manuals for the seat elements concerned.

Paragraphs in this document that are relevant to your safety are marked with a safety symbol.

## Product description

### Checking the wheelchair on delivery

Before you start using it, check that the wheelchair delivered matches the product description below. If you notice any damage caused during transit, please inform your dealer. Life & Mobility trusts that the Canto Nxt will meet your expectations.

### Product description

1. Push handle
2. Operating lever for seat angle
3. Backrest
4. Side equipped with a height-adjustable armrest and padded side cushion
5. Seat
6. Footrest
7. Brake
8. Tilting frame
9. Front wheel
10. Back wheel
11. Anti-tipping supports\*
12. Wheel plate
13. Telescopic tube
14. Backrest hinge
15. Front and back taxi hooks on both left and right

#### Not illustrated:

- - Headrest (optional)
- - Operating lever for backrest angle adjustment (optional)
- - User manual

Push wheelchair

Manual wheelchair

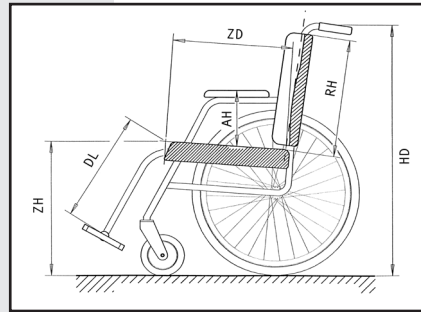
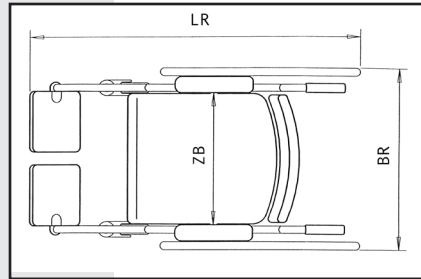
The standard products (standard legrest, standard seat, etc) are described in these instructions for use. There are, of course, many other possibilities, such as the Matrixx seating system, which are given under 'optional'.

\* Anti-tipping supports for preventive safety are supplied as a standard feature with the Canto Nxt

## Technical specifications

- Seat width of push wheelchair and manual wheelchair: 37, 42, 47 or 52 +/- a maximum of 4 cm (continuously adjustable by means of armrests + frame)
- Seat width of push wheelchair in combination with the Kelvin seat element: 34, 40, 46 cm
- Seat depth of push wheelchair and manual wheelchair: 42 - 56 cm (continuously adjustable by means of backrest frame)
- Seat depth of push wheelchair in combination with the Kelvin seat element: 40 - 56 cm\*
- Seat height of push wheelchair: 42, 47, 52, 57 cm (based on 6" and/or 8" front wheels in combination with 121/2" back wheels)
- Seat height manual wheelchair 42, 44.5, 47, 49.5, 52, 54.5, 57 cm (on the basis of 6" and/or 8" front wheels in combination with 20/22/24/26" back wheels)
- Seat height of push wheelchair in combination with the Kelvin seat element: 51, 56 cm (based on 200x42 front wheels in combination with 121/2" back wheels)
- Width of push wheelchair = seat width + 21 cm
- Width of manual wheelchair (in combination with hub brakes with brake levers) = seat width + 28 cm
- Width of manual wheelchair (in combination with tyre brakes) = seat width + 26 cm
- Width of push wheelchair in combination with the Kelvin seat element: 63, 63, 68 cm\*
- Total weight of the basic model = 33 kg
- Weight of heaviest component of the basic model = 22 kg

- Maximum weight of user = 150 kg (Push + manual wheelchairs)
- Maximum weight of user = 125 kg (Push wheelchair in combination with Kelvin seat element)
- Seat angle range = 30 degrees (0-30 or -5-25)
- Back angle range = 0-180 degrees (one adjustment)
- Back angle range of push and manual wheelchair = 85-125 degrees (continuously adjustable)

Life & Mobility recommends using the Canto Nxt in an ambient temperature of between -5° C and +40° C.

### Tyres

The correct tyre pressure is shown on the tyres.

### Expected service life

Life & Mobility assigns a service life of 7 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

\* Depending of the type of seat element.

\*\* Seat angle range in combination with the continuously adjustable backrest angle adjuster: 22 degrees (0-22 / -5-17).

## Optional

In addition to the standard components for the Canto Nxt, the following optional components are also available.

### Legrests

- Comfort legrest
- Central legrest
- Plaster cast legrest
- Amputation support

### Backrest

- Standard backrest (with torso supports)
- Soft backrest
- Anatomical backrest
- Comform backrest
- Invito backrest
- Comfit backrest
- Tracix backrest
- Bendix backrest

### Seat\*

- Memory foam seat
- Plank seat (with rim)
- Contour seat
- Contour+ seat
- Flap seat

### Seat element

- Kelvin seat element
- Flat seat cushion
- Flat seat cushion with abduction pad
- Classic leg support
- Swing leg support

\* All seats are equipped with a recess for the ischial bone.

### Headrest

- Standard headrest small
- Flexible headrest
- Snellius headrest
- Snellius headrest with Edison headrest adapter
- Kelvin headrest small with Edison headrest adapter

### Brakes

#### Push wheelchair

- Central footbrake equipped with 2 brake levers
- Drum brakes operated by accompanying person

#### Manual wheelchair

- Tyre brakes - standard
- Tyre brakes - extended
- Single-hand tyre brake system

### Worktops

- Perspex worktop
- Wooden worktop
- College worktop
- Elbow cushion

### Accessories

- Support plates for footrests
- Abduction block
- Seat belt
- Hoop cover
- Spoke protector
- Matrixx Cleaner

### Electrical options

#### Push wheelchair

- Aura push support

#### Manual wheelchair

- @Work auxiliary drive

## Safety instructions

### Life & Mobility accepts no liability for damage or injury caused by negligence or non-compliance with the safety instructions.

Dependent on the specific circumstances or accessories used, supplementary safety instructions can be required.

Allways notify Life & Mobility and the relevant authority in case of a serious incident.

### Assembly and adjustment

- Before using the wheelchair, check that the components of the wheelchair, such as the sides with armrests and the legrests, are properly fitted and that the seat and backrest, for example, are securely fastened.
- Make sure that the push handles, the armrests, the side cushions, the lower leg length and the headrest, if present, are set to the right height. The legrests must be at least 5 cm off the floor.
- Check the dimensions and set the seat depth and width to the optimum position using the backrest frame and the armrests respectively, if necessary.

### Sitting in the wheelchair

- Make sure that the user's fingers do not become trapped between the frame and the wheels or between the spokes on the wheels. The user or accompanying person should therefore be vigilant in the following situations:
  - If the wheelchair is tilted too far backwards.
  - If the user is sitting lopsidedly in the wheelchair
  - If the user's arms are hanging down limply over the armrest.
  - In any combination of the above situations.

### Use of the wheelchair

(see also page 24, 25 and 26 for more information)

- **N.B.:N.B.:** When using the chair on a slope, the centre of gravity will change and the wheelchair can tip over backwards more easily.
- When propelling the chair manually, be careful of your hands if there is not much space to the left and right of the chair.
- Accompanying persons must never leave the wheelchair on a slope without engaging the brake.
- Never tackle obstacles higher than 10 cm on your own. Only experienced users with sufficient strength are generally capable of tackling obstacles up to 10 cm in height.
- Engage the brake on the wheelchair before getting out of the chair or transferring from it.
- Only use the brake to lock the stationary wheels. The brake is not suitable for slowing the wheelchair down.

### Transporting the wheelchair

(see also page 18 for more information)

- Fix the wheelchair in a taxibus with a four-point securing system.
- Use a three-point safety belt for the transport of the wheelchair with user.

**N.B.:** the wheelchair seat belt is not suitable for use as a safety belt.

During transport of a wheelchair with user, the wheelchair must be fitted with a Matrixx headrest.

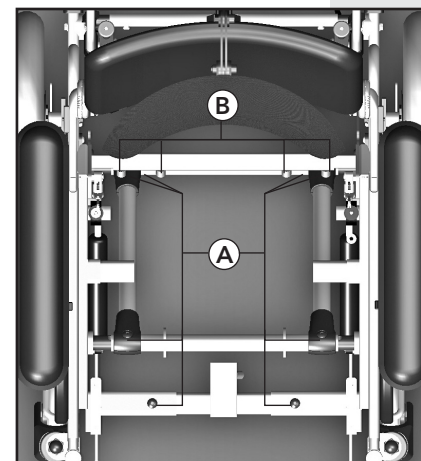
## Settings and adjustments

### Slopes and other obstacles

(see also page 25 for more information)

- The wheelchair is more likely to tip over on a slope than on a level surface.
- Always go off a pavement or step backwards to prevent the wheelchair from leaning over forwards and the user from falling out of it.
- Never go down stairs with the wheelchair.
- Never push the wheelchair up a slope with an angle greater than 5°.
- Ask somebody to help you if you are not sure whether you can reach the top of the slope by yourself.
- Lean or push your body in the direction of the top of the slope. This will shift the centre of gravity of the wheelchair to a better position and will reduce the risk of the chair tipping over backwards.
- Do not push the wheelchair up slopes while the seat is tilted.
- The person pushing the wheelchair must continue to push the chair to prevent it from starting to move under its own momentum.
- Do not turn the wheelchair round on a steep slope.
- If the wheelchair is facing across a steep slope, it can very easily tip over.
- Always keep your speed under control when going downhill.
- The accompanying person must reduce the speed of the wheelchair via the push-handles.
- If you are operating the wheelchair yourself, only slow the wheelchair down by hand, not by using the brake, as this can cause you to lose control of the wheelchair.


Adjust frame width

The Canto Nxt has a wide range of settings and adjustment possibilities with which the wheelchair can be adjusted to suit all the requirements of the user. Life & Mobility recommends that you consult your dealer with regard to the settings. You can easily make adjustments yourself, without the use of tools.

### Frame
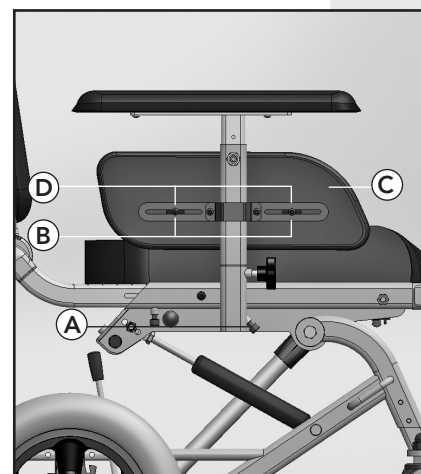
#### Frame width

- Remove the 8 socket screws (A).
- Unscrew the 4 socket screws (B) a few turns.
- Adjust the desired seat width symmetrically.
- Tighten all socket screws again.

#### Seat width

The width of the seat frame is adjustable and can be set to widths of 37, 42, 47 and 52 cm. It is also possible to fit the sides, with armrests, further in or out. This adjustment can be made separately on the left and on the right. One side fitted with an armrest can be moved in and out independently of the other.\*

#### Adjust inward or outward

- Loosen the socket screw (A).
- Set the desired narrowing of the seat width.
- Tighten the socket screw (A) again.
- Repeat this on the other side.

#### Adjust 15 mm extra inward

- Loosen socket screws (B).
- Hold padded section (C) and position the two spacers (D) between the padded section and the metal brace.
- Tighten the socket screws (B) again.
- Repeat this on the other side.

\* In the case of a manual wheelchair, the wheel size and seat angle adjustment might be a hindrance.

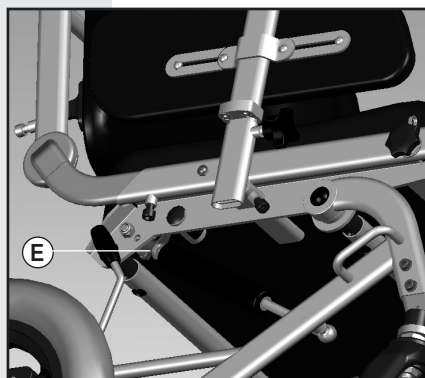
### Seat depth

The backrest frame is attached to the side sections of the seat frame in such a way that it can be slid in and out. The seat depth can be continuously adjusted by moving the backrest frame forwards or backwards.

- Loosen the socket screws (E).
- Slide the backrest frame in or out until you have reached the desired seat depth.
- Determine the precise size using the graduation marks on both sides of the seat frame.

Adjust seat depth

**N.B.:** make sure that the frame is straight, that is, that the graduation marks on both sides correspond.


- Retighten the socket screws.

In the case of deeper seats the centre of gravity is further back in relation to the back wheels. Readjust the ride characteristics, if necessary (see below).

### Ride characteristics

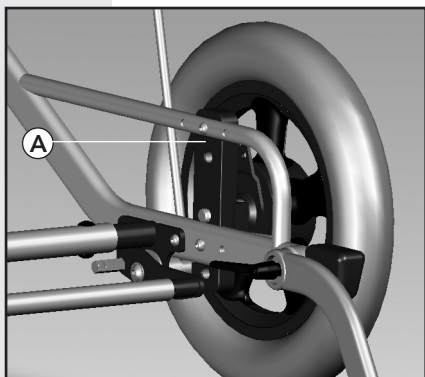
The ride characteristics of the Canto Nxt are adjustable.

- Loosen the socket screws (A).
- Move the wheel and wheel plate forwards\* or backwards (the back axle hole for more stability and the front axle hole for a more comfortable ride; the ride characteristics are related to the seat depth).
- Retighten the socket screws.
- If your Canto Nxt is a manual wheelchair that has tyre brakes, the brakes must be adjusted.


Adjust riding features

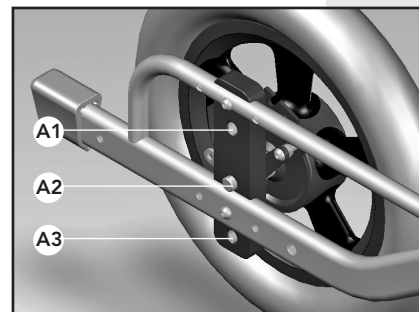\* Not possible in the case of a push wheelchair.


Push wheelchair

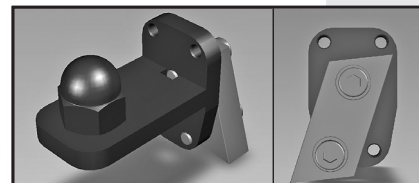T1-A

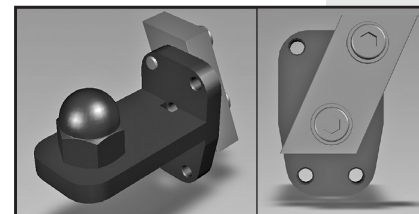T1-A

T1-B

T1-B

In connection with preventive safety, for the Kelvin, and where the backrest is continuously adjustable, the rear wheels are mounted in the rearmost position in the factory.

### Seat height

You can adjust the seat height to various heights depending on whether your Canto Nxt is a push or manual wheelchair. There are 3 axle holes and 3 different telescopic tubes available for adjusting the seat height. For the manual wheelchair, the seat height is related to the wheel size.

**N.B.:** seat angles can differ with the different Matrixx seats.

### Push wheelchair

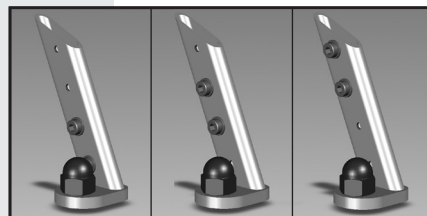
Required seat height	Tyre size	Position
42 cm*	6" x 12,5"	A1 / T1-A
47 cm	6" x 12,5"	A1 / T2-B
	8" x 12,5"	A1 / T1-B
52 cm	6" x 12,5"	A2 / T3-A
	8" x 12,5"	A2 / T2-B
57 cm	8" x 12,5"	A3 / T3-A

\* At a seat height of 42 cm, adjusting the wheelchair to a neutral seat angle is realised by placing the gas springs in the upper frame hole, see also page 12.

*Push wheelchair + Matrixx structure*


Required seat height	Tyre size	Position
50 cm*	6" x 12,5"	A1 / T1-A
55 cm	6" x 12,5"	A1 / T2-B
55 cm	8" x 12,5"	A2 / T1-B

T2-A T2-B T2-C

*Push wheelchair + Kelvin seat element*

Required seat height	Tyre size	Position
51 cm	200 x 42 x 12,5"	A1 / T1-B
56 cm	200 x 42 x 12,5"	A1 / T2-B

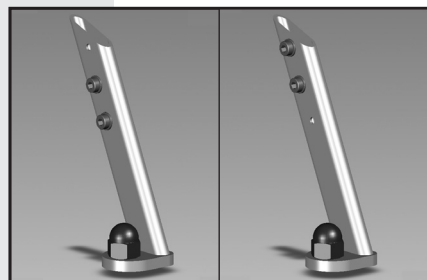
T3-A T3-B

*Manual wheelchair*

Required seat height	Tyre size	Position
42 cm*	20" x 6"	A1 / T1-A
44,5 cm	20" x 6"	A1 / T2-A
47 cm	22" x 6"	A1 / T2-B
	22" x 8"	A1 / T1-B
49,5 cm	22" x 6"	A2 / T2-C
	22" x 8"	A2 / T2-A
52 cm	24" x 6"	A2 / T3-A
	24" x 8"	A2 / T2-B
54,5 cm	24" x 6"	A3 / T3-B
	24" x 8"	A3 / T2-C
57 cm	26" x 8"	A3 / T3-A

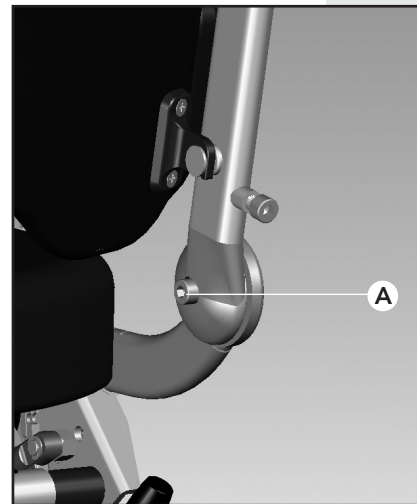Manual wheelchair

*Backrest angle (adjustment)*

- Loosen the socket screws (A) in the backrest hinge points.
- Set the backrest to the required angle.
- Then retighten the socket screws.

**! N.B.:** make sure the left and the right push bars are at the same angle with the aid of the graduation mark.

*Backrest angle (continuously adjustable)*

The backrest angle can be adjusted using the central gas spring and the operating lever on the left-hand push-handle.

- Squeeze the left-hand operating lever to disconnect the gas spring.
- Adjust the backrest to the desired angle.
- Release the lever.

Left-hand push-handle (red sticker)

*Seat angle*


The seat angle can be adjusted using the 2 gas springs and the operating lever near the push-handle.

- Press the operating levers to disengage the gas springs.
- Set the seat angle as required.
- Release the operating levers.

*Seat angle range*

The seat angle range is 30°. It can be adjusted from 0° to 30° and from -5° to 25°.

Required seat height	Seat angle	Position
42 cm	0° tot 30°	G1
	-5° tot 25°	G2
≥ 44,5 cm	0° tot 30°	G2
	-5° tot 25°	G3

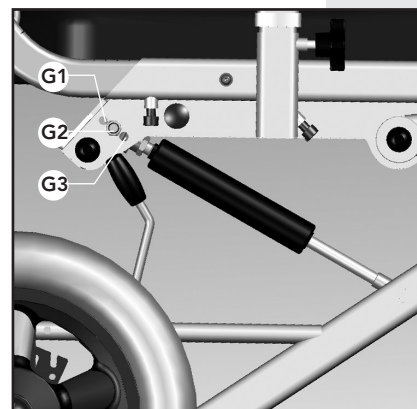\* At a seat height of 42 cm, adjusting the wheelchair to a neutral seat angle is realised by placing the gas springs in the upper frame hole, see also page 12.

**N.B.:** seat angle range in combination with the continuously adjustable backrest angle adjuster: 0° to 22° or -5° to 17°.

### Armrests

The height, depth and width of the armrests can be adjusted.

#### Adjusting the height

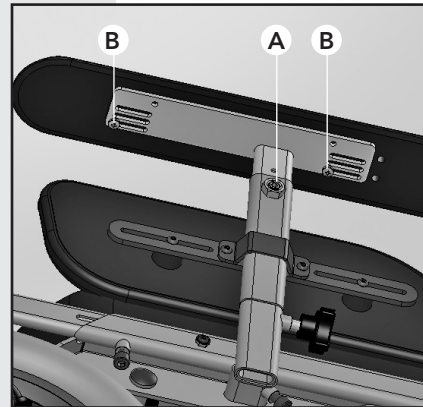
- Unscrew the adjustable screw (A) at the side of the armrest a couple of turns.
- Set the armrest to the required height.
- Retighten the star knob.

Right-hand push-handle (green sticker)

#### Adjusting the depth

- Loosen the screws (B) fully (different hole in armrest) or partially (using the slot).
- Set the armrest to the required depth.
- Retighten the socket screws.

#### Adjusting the width

- Loosen the screws (B) fully.
- Set the armrest to the required width.
- Tighten the screws in the required press-in nut and slot of the armrest.

### Side cushions

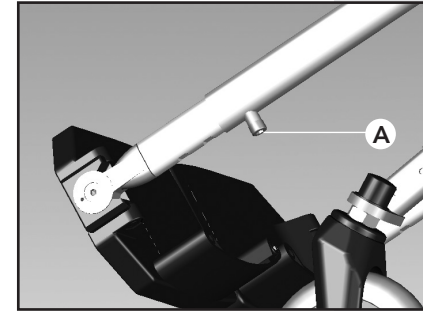
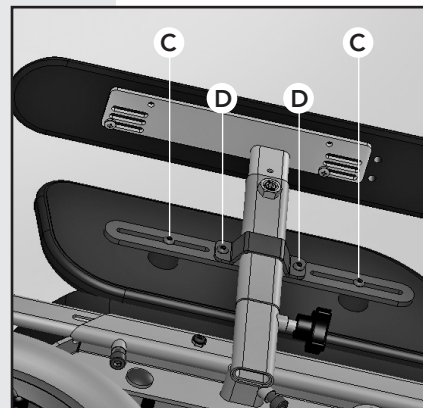
The height and depth of the side cushions are adjustable.

#### Adjusting the depth

- Loosen the socket screws (C).
- Set the side cushions to the required depth.
- Retighten the socket screws.

#### Adjusting the height

- Loosen the socket screws (D).
- Set the side cushions to the required height.
- Retighten the socket screws.

### Seat width

Follow the instructions on page 15 to adjust the seat width.

#### Lower leg length

- Loosen the socket screw (A) one turn.
- Set the legrest to the required lower leg length.

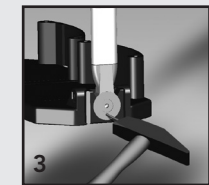
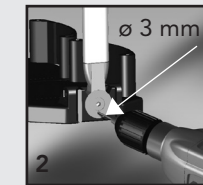
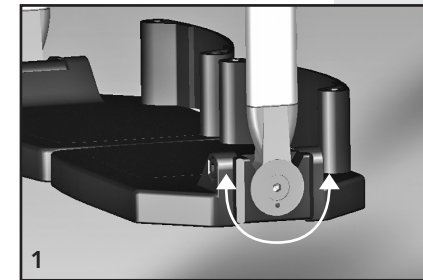
**N.B.:** make sure the legrest is at least 5 cm off the floor.

- Retighten the socket screw.

#### Fixing the ankle hinge

- Set the ankle angle to the desired position.
- Drill a hole in the hinge.
- Fix the ankle hinge in place by means of a fixing pin.

#### Adjusting the push-handle

- Loosen the butterfly knob one turn.
- Adjust the handle to the required height.
- Retighten the butterfly knob.

## Making the wheelchair smaller

### Anti-tipping supports

The anti-tipping supports can be adjusted using the lever (A). The recommended height is 3 cm from the floor.

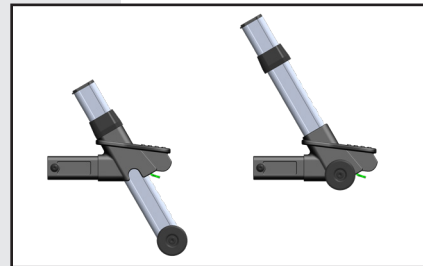
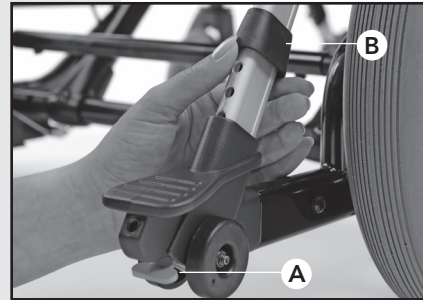
#### Please note:

- Make sure that the pin clicks properly into the opening.
- Inform the user if the anti-tip support is folded away.

#### Tip:


- The sliding sleeve can be used to mark a standard position.

The anti-tip support - if properly adjusted - protects the wheelchair user from tipping backwards. The risk of tipping the wheelchair backwards is influenced by:

- the position of the user's center of gravity in the wheelchair (which is determined by factors such as axle position, backrest angle, seat angle, seat tilt, and weight distribution over the body (e.g., amputations)).
- any restlessness or spasms of the user.
- the slope on which you are driving.

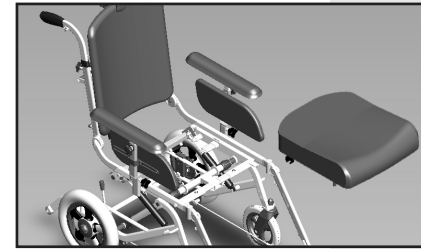Adjust the anti-tip as necessary based on the above considerations.

### Removing the seat (see fig. 1)

- Pull up the back of the seat.
- Lift the seat out of the frame.

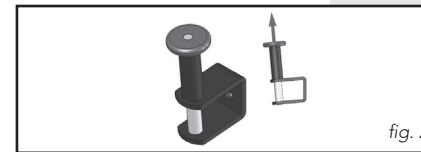
fig. 2

### Remove the backrest from the backrest frame (see fig. 2)

- Lift the left and right knobs of the top fittings at the same time.
- Rotate the backrest forward so that the U profile releases from the backrest attachment points.
- Release the knobs of the top fittings.
- Lift out the backrest.

fig. 3

### Removing the legrests (see fig. 3)

1. Remove the leg rests from the top.

### Side equipped with armrest (see fig. 4)

- Loosen the star knob at the bottom of the sides.
- Lift the entire side upwards out of the seat frame.

### Removing the headrest (see fig. 5)\*

- Loosen the star knob.
- Remove the headrest from the wheelchair.

### Backrest tubes (see fig. 6)

- Loosen the socket screws in the backrest hinge points.
- Move the backrest frame towards the seat frame.
- Retighten the socket screws.

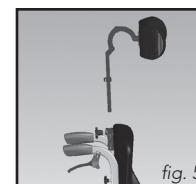
fig. 5

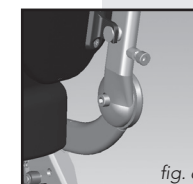
fig. 6

\* If applicable

## Use

### Transferring to and from the chair

The Canto Nxt is a wheelchair for users with various handicaps. Transfers from the wheelchair to, for example, a bed, toilet or shower and vice versa can be made in various ways. The extent to which the wheelchair user is able to cooperate determines how he or she should be lifted. Both active and passive transfers are possible with the Canto Nxt, with the aid of the care provider or using a lift.

**N.B.:** always engage the brake before transferring to or from the chair.

#### Forwards transfer

- Fold up the footrests or fold away the legrests.
- Set the chair angle at a negative value, if necessary.

#### Sideways transfer

Remove the sides and armrests, if required.

**Tip:** during a transfer from a passive lift into a Canto Nxt, first tip the wheelchair without user so that the user can be optimally positioned in the chair.

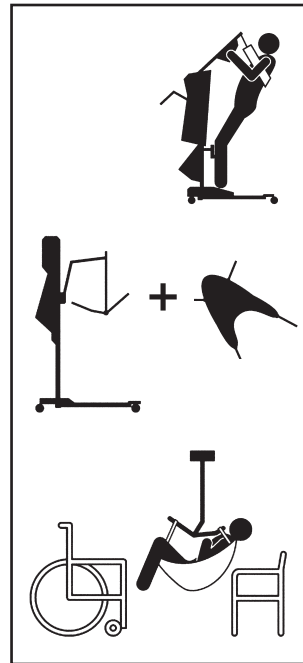
### Tackling obstacles

- Press the push-handles downwards and push with your foot on the step tube.
- Push the wheelchair onto the obstacle and remove your foot from the step tube.
- Push the wheelchair further and lift the back wheels over the obstacle by the push-handles.


You can go up steps forwards but it is better to go down steps backwards.

**N.B.:** keep your speed under control on gentle slopes.


Stairs

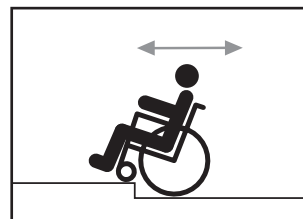Pavement of step

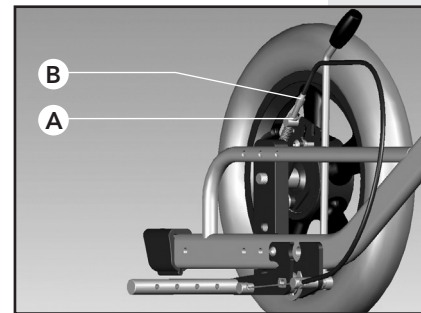Gentle slope

Adjusting the brakes

### Brakes

The brakes of the Canto Nxt are correctly adjusted ex factory. If your Canto Nxt is a power wheelchair, it has drum brakes with a brake lever. The central foot brake is located at the bottom of the wheelchair at the back. When the bracket is up the wheelchair brake is disengaged; when it is down the brake is engaged.

It may be the case that over the years the braking power decreases or is no longer uniform for both wheels. You should then adjust the brakes.

- Loosen the locknut (A).
- Turn the adjustment bolt (B) outwards\*.
- Check that the braking power is adequate.
- Adjust it again if necessary.
- Tighten the locknut (A).
- Repeat this on the other side, if required.

\* If you turn the adjustment bolt too far outwards, the brake will be engaged in the neutral position too. The wheels must turn freely in the neutral position.

## Transporting the Canto Nxt

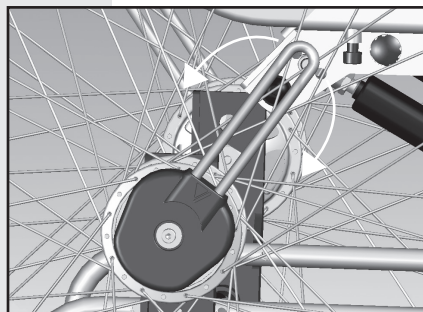
### Manual wheelchair

If you have a manual Canto Nxt, the brake is integrated in the hub of the wheel with handrim. In addition, each wheel can be braked forwards or backwards by means of moving the brake lever.


Intensive or long-term use of the brake may reduce the braking power. You should then adjust the brakes.

Use a (3 mm) Allen key to turn the set screw underneath the brake lever half a turn.

Check that the braking power is now adequate. Adjust it again if necessary.

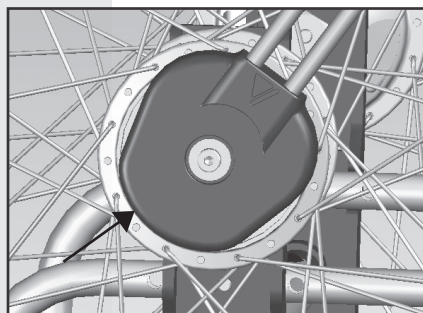Brake of manual wheelchair in neutral position

Adjustment screw

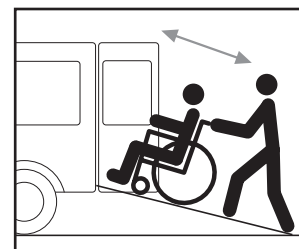
Figure 1

### Taxibus

The Canto Nxt was designed to be secured in a forward direction in a taxi van. The Canto Nxt was tested positively as a crash-test safe wheelchair in accordance with ISO 7176-19:2008. The Canto Nxt can therefore be transported as a chair with an occupant in a taxi van (see also the designation sticker on the chair).

To ensure safe transport, however, the condition of the occupant and the doctor's recommendation are decisive. Life & Mobility still recommends that the occupant should transfer from the Canto Nxt to a fixed seat, if possible.

- The Canto Nxt was dynamically tested in a forward direction, where the test dummy was secured in the taxi van with a hip belt and a shoulder belt. Its suitability during other collisions (such as a lateral collision) or other types of belts was not tested.
- The Canto Nxt was tested in the various configurations. The critical configurations with Kelvin seating elements and Matrixx headrests are also 'crash-test safe'.
- The chair should be fitted with a Matrixx headrest.
- The Canto Nxt should be attached to the floor of the taxi using a four-point fastening system in accordance with EN 10542-2.

**N.B.:** if there is no crashtest safe sticker on the wheelchair, it is not a wheelchair which can be transported safely.

**N.B.:** the Canto Nxt frame is only crash test safe in combination with Matrixx seat elements and Matrixx headrests.

- The positions of the four attachment points on the Canto Nxt are indicated with a hook symbol sticker (see figure 2).
- Easy access and manoeuvrability in motor vehicles can be significantly influenced by wheelchair size and agility. Smaller wheelchairs with a shorter turning circle generally offer better accessibility to the vehicle and manoeuvrability to a forward position. The Canto Nxt was assessed positively for securability according to ISO 7176-19:2008, Annex D.
- The Canto Nxt should be fastened using anchoring systems that are suitable for the 4 attachment points.
- The user must wear a three-point safety belt in the taxibus. These safety belts must be attached to the taxibus.
- The lap belt must be positioned low across the user's waist (see figure 3). The angle between the belt and the floor must be between 30° and 75° (a bigger angle is better).
- Pelvic restraints should make full contact across the front of the body near the junction of the thigh and pelvis. The upper belt must fully support the user as illustrated in figure 4.
- The diagonal belt must be positioned across the user's shoulder and chest as shown in figure 5.
- The belt must be sufficiently tight but not uncomfortable for the user.
- The belt must not be twisted.
- The belt must not be resting against armchair components (like armrests or wheels) as a result of which it is no longer fully supporting the body of the user (see figure 6).
- If possible, the Canto Nxt should not be transported in a tilted-back position.
- The Canto Nxt must be transported with the user facing forward, with a securing system that complies with the instructions of the manufacturer of the securing system.

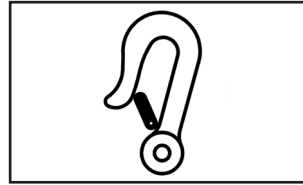
Figure 2

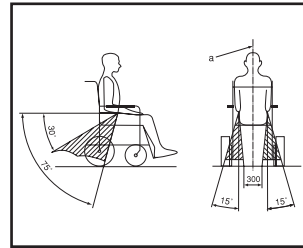
Figure 3

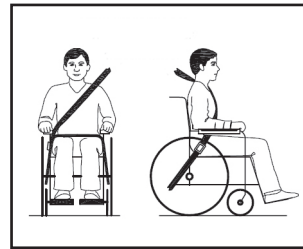
Figure 4

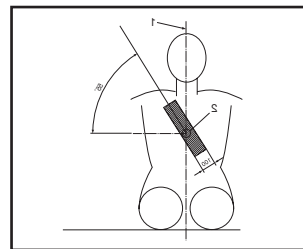
Figure 5

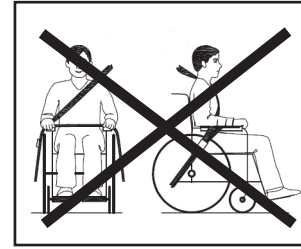
Figure 6

- The three-point safety belt is intended to avoid injury to the head and chest.
- If a worktop is fitted, it must be removed in order to prevent injury to other passengers. The worktop should be safely stowed away elsewhere in the vehicle.
- If possible, any accessories must be secured or removed.
- Following an accident, the wheelchair must be assessed by a representative of the manufacturer to ensure that it can still be used.
- Only lift the wheelchair using fixed frame components and not removable accessories such as armrests, legrests seating, back support etc.
- The attachment points, structural components and frame components must not be modified or replaced without first consulting the manufacturer

#### Seat belt

The seat belt is intended to keep the user in position. Attach it on or around the metal seat frame.

**N.B.:** the optional wheelchair seat belt is not suitable for use as a safety belt

#### Car

In addition to a taxibus, the Canto Nxt can also be transported folded up in the boot of an estate car. Read on page 15 how to fold up the wheelchair. The wheelchair can be lifted into the car by two adults.

## Warnings

- In accordance with the agreement, the Canto Nxt base was declared collision-test safe (ISO 7176-19:2008) in combination with specific orthoses and was issued with CE marking.
- The Canto Nxt has been dynamically tested in a forward direction, where the test dummy (= 103 kg) was secured with a hip belt and a shoulder belt.
- Secure the chair in a taxi van with a 4-point fastening system. Also use a 3-point safety belt for transporting the wheelchair and occupant to reduce the risk of head and chest injuries.
- The sitting posture belt is unsuitable as a safety belt.
- Worktops should be removed and stored safely in the taxi.
- All accessories should be secured or removed and safely stored in the taxi.
- After every accident, the manufacturer's representative should examine the wheelchair to determine whether it can be used again.
- The fastening points or structural and frame components should never be changed and/or replaced without first consulting the manufacturer.
- Any batteries should be "spill-proof".
- The belt clasp must be positioned so that no wheelchair parts can come in contact with it during a collision.

## Minor maintenance (every two weeks)

You can do minor maintenance on the chair yourself. Clean your wheelchair with a damp cloth, water and an all purpose cleaner.

**N.B.:** never use abrasive and aggressive cleaners or solvents (thinners, benzine or similar).

The gas springs and the gas spring block (the rectangular block at the end of the gas spring) should be lubricated regularly with, for example, Teflon spray. This will keep them working properly. Check that all fastenings are secure. Tighten if necessary.

**N.B.:** over-tightening the screws can damage the wheelchair.

Check the tyre pressure and condition of the tyres (ageing) regularly. Soft tyres make the wheelchair more difficult to use as they increase resistance. Use, if necessary, the valve nipple supplied with the wheelchair for pumping up the tyres (for example, when using a bicycle pump). The correct tyre pressure is shown on the tyres.

Check the wheel tension on a regular basis (once per quarter, depending on the intensity of use) to make sure that your wheelchair continues to ride comfortably and safely. Ask your dealer for assistance if necessary.

## Major maintenance (annual)

Your dealer will be happy to carry out the major maintenance of your wheelchair. You can then be sure that you will be getting the best possible use out of your wheelchair. You should also consult your dealer if components need replacing.

## Technical problems

In the unlikely event that you experience technical problems with your wheelchair, please contact your dealer. The dealer will decide whether he can repair the chair or whether it will have to be returned to the factory.

You may be able to solve some problems yourself.

*The seat angle can no longer be adjusted*

- Check whether the operating cable is bent or has become trapped somewhere.
- Check whether the gas spring block moves when the handle is pressed. Lubricate the gas spring block if necessary.

*Movable components such as the push-bar, height-adjustable armrest, legrest and sides get stuck*

- These components can become jammed in the frame if too much pressure is exerted on them. Carefully tap the components loose with a plastic hammer.

*Socket screws for one-off adjustment cannot be loosened*

- The screws may have been over-tightened. Try placing a tube over the long end of the Allen key so as to lengthen the turning arm. Now try loosening the screw.
- The socket head may be stripped: Try gripping the screw with a monkey wrench. If this fails consult your dealer.

*The brakes have insufficient braking power or do not brake uniformly on both sides*

- You should adjust the brakes; go to page 25.

**N.B.:** all repairs other than those described above must always be performed by an authorised dealer. Improper repairs may endanger your health. Furthermore, repairs carried out by unauthorised persons will invalidate the guarantee on your chair. Use your common sense and decide whether you are in a position to resolve technical problems on the wheelchair yourself. If in doubt, always consult your dealer. When adjusting and installing or removing parts, consult this manual and follow your dealer's advice. Do not place yourself or the wheelchair user in danger.

## Guarantee

### Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product. Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The guarantee does not cover faults caused by: failure to comply with instructions for use and maintenance instructions, abnormal use, wear and tear, negligence, overloading, accidents caused by third parties, use of non original components or faults the cause of which is not related to the product.

This guarantee supersedes any other guarantee either laid down in law or verbally notified, with the exception of any written guarantee provided by Life & Mobility.

Guarantees are only valid within the EU.

This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication.

The information in this manual applies to the standard version of the product. Life & Mobility does not accept responsibility for any damage that may occur due to changes or modifications to the product after selling it for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.

The information in this manual may not be reproduced for commercial goals and/or made public by any means without prior written permission from Life & Mobility.

Pursuant to the Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.

No rights can be derived from this text. Subject to changes.

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© Life & Mobility – All rights reserved