MATCH

AURA

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Safety instructions	4
Introduction	11
Technical specifications	12
Product description	14
Use	15
Maintenance	23
Transport & storage	24
Faults	27
Warranty	32
Disposal of product & packaging	33

Safety instructions

Attention:

- This manual is supplemental to the wheelchair user manual. All safety instructions from the wheelchair user manual also apply to a wheelchair fitted with an Aura push assist.
- Read this user manual and the wheelchair user manual carefully before using the push assist.
- If you see this symbol in the manual, it indicates an instruction. Always follow this instruction. Negligence may adversely affect the operation of the Aura push assist and may cause physical injury or damage to the push assist, the wheelchair or the environment.

General

Warning:

- Never change the construction of the push assist.
- Never try to open the housing of the electronic components. This could cause irreversible damage and you will void the warranty.

Attention:

- The Aura push assist has a fully programmable electronic control system. Changes to programming must only be made by Life & Mobility authorised service engineers and dealers.
- Only use original Life & Mobility parts and accessories.
- Never cover or remove the safety instructions and stickers.

Safety instructions

Personal requirements

The Aura push assist is an assist for the companion and places no additional demands on the wheelchair user. The user requirements in the wheelchair user manual remain in force.

Attention:

- In the event of fault or an empty battery, the push assist will be disabled. In such circumstances, the companion needs to be capable of making the wheelchair safe. This can be done for example by setting the wheelchair in free-wheel and continuing to push manually or by applying the brake and seeking assistance.
- When using the Aura push assist on slopes, the fitness of the companion determines in part the maximum slope on which the wheelchair may be used safely.

Assembly and adjustment

Warning:

- Before using the push assist, check that all its parts are properly mounted and secured.

Attention:

- Check that the wheelchair has the correct balance setting before using the wheelchair for the first time with the Aura push support. The centre of balance may have moved backwards due to the extra weight of the battery and the greater speed of acceleration.
- On setting the height and angle of the push handles, check that the cables cannot become trapped.

Safety instructions

Using the Aura

(see also the chapter 'Use').

Warning:

- Switch off the push assist if the wheelchair is left without supervision.

Attention:

- It is compulsory in the Netherlands to at least have WA liability insurance for your electric push support.*

Slopes and other obstacles

Warning:

- Before releasing the parking brakes using the free-wheel lever, check that the wheelchair cannot make any unwanted movements.
- This can lead to dangerous situations, particularly on slopes. Do not switch to free-wheel unless strictly necessary. In these kinds of situations allow the user to step out first and ensure that there is sufficient help in order to keep hold of the wheelchair. (See also 'Using without push assist').

Attention:

- Avoid slopes with a gradient greater than 7°.
- When going down a slope, always keep your speed under control.
- The Aura push assist uses much more electricity going up a slope than on a flat surface. This can considerably reduce the operating range. Pay attention to the battery indicator in order to prevent unexpected stoppages.

*WA is liability insurance required by government legislation. Any damage you cause to others with your push support, and for which you are legally liable, will be reimbursed. This does not apply to damage to your wheelchair that is your own fault. An insured vehicle is identified by an insurance plate at the rear.

Safety instructions

Battery charger

(See also the chapter 'Use').

Warning:

- Only use the battery charger to charge the Aura push support 36V Li-ion/Li-Fe rechargeable battery. Charging batteries other than those supplied with the Aura push support may cause explosions and bodily injury.
- Charging non-rechargeable batteries can cause explosions.
- Short circuiting the + and - of the battery charger's plug may cause shocks, fire and/or other accidents.
- Never charge the Aura push support with someone sitting in the wheelchair.
- The operation of the battery charger is reserved for those who have been able to fully understand the instructions and safety precautions.
- Keep the battery charger out of reach of children.
- Never open the battery charger.
- Servicing must only be performed by certified service technicians.
- Never leave the battery or wheelchair unattended while charging.

Be careful:

- Always connect the charging cable first to the Aura and then plug the battery charger into the socket.
- Stop charging by removing the charging cable from the socket:
 - if charging is not completed within the prescribed maximum of 7 hours.
 - if there is an abnormal smell, leaks, or deformation of the charger, battery or other parts.

Safety instructions

Attention:

- The battery charger will heat up slightly during charging. This is normal.
- Check the insulation of the plug and cable every time before use.
- Do not use any cables other than the one supplied.
- Remove the plug from the socket after completion of the charging cycle.
- Use the battery charger only in well-ventilated areas.
- Do not expose the battery charger to rain and moisture.
- Never cover the battery charger and keep it away from other objects.
- Always keep the battery charger horizontal (on the rubber soles).

Battery

Warning:

- If there is an unusual smell, smoke, leak and/or deformation of the battery, immediately discontinue use.
- Do not insert (metal) parts into the charging socket, this may cause a short circuit.

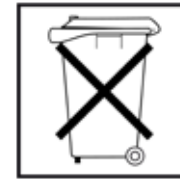
Be careful:

- Make sure you do not overload the Aura. Observe the maximum user weight and operating conditions.
- Turn off the Aura immediately if you notice any defects in the battery or electrical components.

Attention:

- Do not use chemical cleaners on the electrical components.
- Make sure that the electrical components are protected against excess water, e.g. while cleaning.

- Do not dispose of empty or defective batteries with normal household waste. Check with the local government for the regulations that apply for the disposal of Household Hazardous Waste.

Transporting the wheelchair without occupant

(see also the chapter 'Transport and storage').

Warning:

- Never remove the push handles as this will damage the cabling.
- Ensure that the control unit and control lever remain free during transportation. These can become damaged if they are knocked or wedged.

Attention:

- The wheelchair can be folded as described in the wheelchair manual.
- The Aura push assist makes the wheelchair's basic section 16 kg heavier than the standard wheelchair. With push assist, the basic section therefore weighs approximately 55 kg.
- Switch off the push assist while transporting the wheelchair without occupant.

Transporting the wheelchair with occupant

(see also the chapter 'Transport and storage').

Warning:

- Transportation with occupant is only permitted if the wheelchair is provided with a crash-test safe sticker.

Attention:

- Switch off the push assist while transporting the wheelchair with occupant.

Safety instructions

Miscellaneous

Be careful:

- The Aura push assist is splash-proof. You should therefore avoid driving through shallow or deep puddles.
- The wheelchair on which the Aura push assist is fitted should never be sprayed with a high-pressure cleaner or hosepipe as this can cause irreversible damage to the Aura push assist.
- Do not allow the Aura push assist to come in contact with seawater as seawater is corrosive and will rust the Aura.
- Do not allow the Aura push assist to come in contact with sand, as it may penetrate into the moving parts of the Aura, causing unnecessarily fast wear and tear.

EMC-approved

The Aura Push assist is approved for EMC (electromagnetic compatibility). Tests have shown that the Aura is not sensitive or damaging to mobile communication equipment and electronically controlled equipment. Despite this, please be careful when using the Aura together with mobile or electronic equipment.

It is also not advised to use the Aura close to strong electric or magnetic fields as can be found in industrial environments. If you notice problems, please switch off the push assist.

Introduction

Dear user,

A wheelchair should be comfortable, not only for the user, but also for the companion. Only then is optimum mobility guaranteed.

Your wheelchair is provided with an Aura push assist. In this user manual you will find everything you need to know about the Aura, from safety instructions and operating functions to maintenance tips. Sections important for your safety are marked in the text with a safety symbol.

Warning:

- Not following the safety directions, instructions, warnings and maintenance recommendations may have a negative effect on the push assist performance and could result in physical injury, or damage to the wheelchair or the environment.

Attention:

- This manual is supplemental to the wheelchair user manual. All safety instructions from the wheelchair user manual also apply to a wheelchair fitted with an Aura push assist.
- Read this user manual and the wheelchair user manual carefully before using the push assist.
- Keep the user manual in a safe place for a possible future user.

We are happy to keep you informed on updates, user experiences and other useful information. You can read about the latest developments regularly on the www.life-mobility.com website.

Kind regards,
Life & Mobility Management and Staff

Technical specifications

The Aura push assist specifications are given below. Other specifications for the wheelchair can be found in the wheelchair user manual.

- Intended use: Indoors and outdoors on hard surface
- Maximum safe gradient: 7 degrees
- Extra weight of Aura push assist: 16 kg
- Maximum speed: 6 km/h
- Battery type: Li Ion, 36 V
- Battery capacity: 11.6 Ah
- Battery weight: 3 kg
- Charging time: 6 hours for full charge
- Operating range: 18 km*
- Charger input: 100 – 240V 50/60Hz
- Charger output: 24V DC/2.35 A
- Environmental temperature during use: -5°C to 45°C
- Environmental temperature during charging: +10°C to +40°C
- Environmental temperature during storage and transportation: -10°C to +60°C**

Test specifications

The Aura push assist in combination with the wheelchair satisfies the following standards:

- EN 12183:2009 Hand-powered wheelchairs
- EN 12184:2009 Electric wheelchairs, the electric part of the wheelchair
- ISO 7176- 8:1998 Static, shock, fatigue and strength requirements
- ISO 7176- 9:2001 Climate tests
- ISO 7176- 14:2008 Control system requirements
- ISO 7176- 19:2001 Wheelchair requirements for use as wheelchair in minibuses.
- ISO 7176- 21:2009 Electromagnetic compatibility (EMC)

* In practice the operating range may be lower. The weight of the user, weather conditions, ground surface, slopes and age of the battery can have a negative influence on the operating range.

** For long-term storage with battery: 10-30°C

CE sticker

Technical specifications

Servicing and technical support

Contact your dealer if you have any questions, or for maintenance or repairs. Always have the serial number ready. You can find this on the mobility scooter's CE sticker (X).

Expected service life

Life & Mobility assigns a service life of 7 years to this product. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

Product description

Use

Checking the delivery

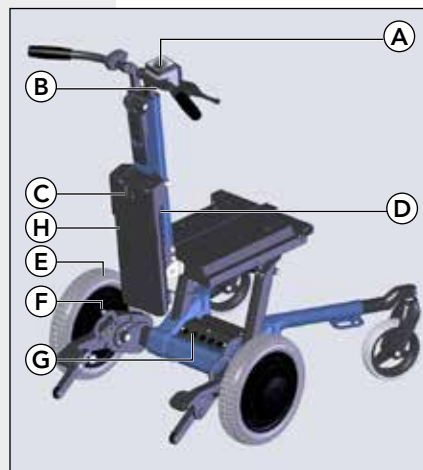
Before using the push assist, check if the delivery matches the product description below. If you find any transport damage on delivery, please inform your dealer.

Life & Mobility trusts that the push assist meets your expectations and hopes it brings you much convenience.

Product description

Push assist


- A. Control unit control panel
- B. Control unit control lever
- C. Charger connection point
- D. Battery switch
- E. Wheel with propulsion motor
- F. Free-wheel lever (small black lever)
- G. Housing for controller and battery
- H. Battery

Product description

Control panel

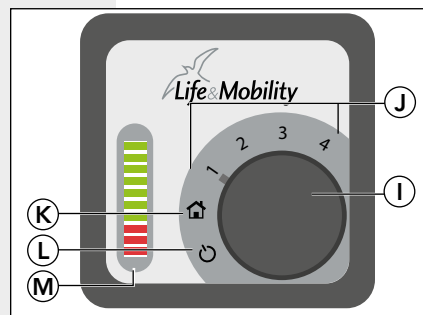- I. Selection switch
- J. Speed setting 1 to 4
- K. Indoor setting
- L. Off position
- M. Battery indicator


Control panel

Not pictured

- Battery charger with connection cable

The battery on delivery

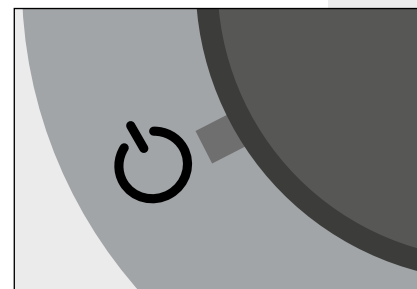The battery of the wheelchair is charged on delivery. This means that the wheelchair is ready to use. It is possible that the battery is not fully charged because it has discharged partially during storage and transportation.


'Off' position

Starting and setting speed

The battery has a switch to turn off the push support, for example, to prevent unintentional use. Turn on the switch to move (the red light on the switch is now on).

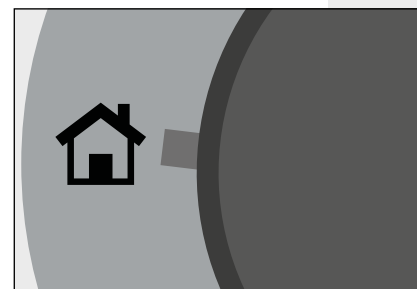The Aura push assist is provided with a selection switch for switching the push assist on and off. You can also select the maximum speed with this switch.


Indoor setting

'Off' position

The push assist is switched off. The wheelchair's parking brake is activated.

Indoor setting

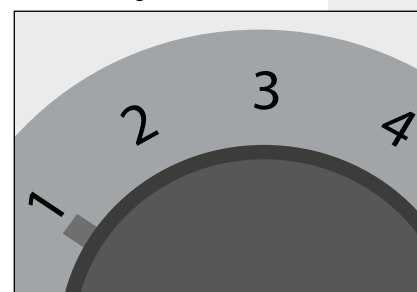The push assist is switched on. The driving speed and acceleration are limited for use indoors and for manoeuvring in small spaces.

Speed position 1

Speed setting 1 to 4

The push assist is switched on. The maximum driving speed can be set from approximately 2 km/hour (position 1) to approximately 6 km/hour (position 4)

Tip: If required, the dealer can adjust the speed and acceleration.

Use

Use

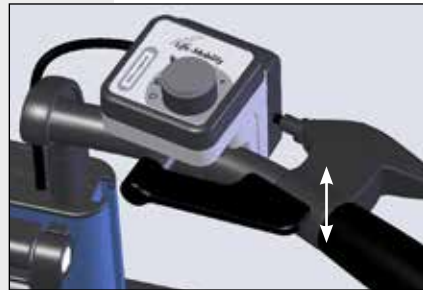
Using the Aura

⚠ Attention: Before making a journey, check whether the battery is fully charged. Insufficient charging can mean that the assist stops on the way (see also 'Checking the battery').

1. Check that the battery switch is on (the red light on the switch is on).
2. Switch on the push assist.
3. Select a maximum driving speed.

Drive forwards

- Push the control lever downwards.
The wheelchair will move forwards.

Operate the control lever

⚠ Warning:

- Never fasten the throttle.

⚠ Attention:

- The further the lever is depressed, the higher the speed.
- The selection switch set-up determines the speed of a fully depressed control lever.
- Release the throttle in good time when approaching obstacles. Moving too fast over an obstacle (e.g. pavement) may cause the wheelchair to suddenly stop and cause discomfort to the occupant.
- The wheelchair parking brake is released automatically if the control lever is depressed. When the control lever is no longer used and wheelchair comes to a halt, the parking brake will again be deployed. Switching the brake on and off makes a clicking sound.

- Avoid getting clothing etc. entangled in the throttle.

Driving in reverse

- Push the control lever upwards.
The wheelchair will move backwards.

⚠ Attention:

- The further the lever is pushed upwards, the higher the speed.
- The maximum reverse speed is approximately half the maximum forward speed.

Brakes

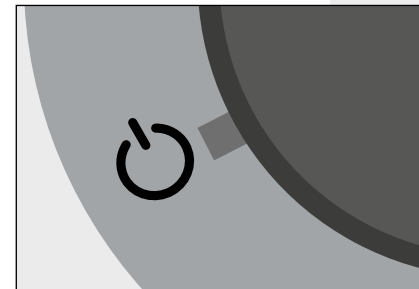
If you need to brake during a walk when using the push assist, the push assist will support this. This makes it easier to walk down a slope.

- Do not depress the control lever as far, or release it completely.
The parking brake will be activated automatically when the throttle is not operated.

Transfer of occupant

⚠ Warning:

- Always switch off the push assist when the occupant gets in or out.

Switch off push assist

Use

Use

Driving without push assist

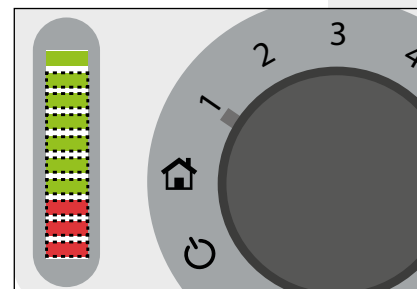
Under normal circumstances a wheelchair with Aura push assist can only be moved with the push assist. When the push assist is switched off, the parking brake is activated and driving is no longer possible.

In order to be able to move the wheelchair if the battery is empty or if there is a fault, the motors are provided with free-wheel levers. These levers can be used to switch off the parking brake when the push assist is switched off.

Free-wheel sticker

Lever in the forward position:
neutral activated
Lever in the backwards position: parking brake activated

Free-wheel lever activated,
push assist on

display nine fast-flashing lamps.

2. To turn off neutral push the lever on both wheels backwards, this automatically activates the parking brake.
3. Briefly turn off the Aura push support and turn it on once more to use it.

Slopes

The Aura push assist makes it significantly easier to drive up and down slopes. However, there are a number of points that need extra attention on slopes.

Attention:

- Avoid slopes with a gradient greater than 7°.
- When going down a slope, always keep your speed under control.
- The Aura push assist uses much more electricity going up a slope than on a flat surface. This can considerably reduce the operating range. Pay attention to the battery indicator in order to prevent unexpected stoppages.
- If it is necessary to brake on a slope by switching off the free-wheel levers, then ensure that you are certain that the wheelchair cannot roll down the slope. See also 'Driving without push assist'.

Warning:

- Before releasing the parking brakes using the free-wheel lever, check that the wheelchair cannot make any unwanted movements.
- This can lead to dangerous situations, particularly on slopes. Do not switch to free-wheel unless strictly necessary. In these kinds of situations allow the occupant to step out first and ensure that there is sufficient help in order to keep hold of the wheelchair.

Attention:

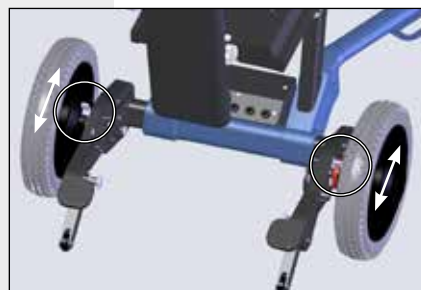
- Ensure that the parking brakes are always switched on again before leaving the wheelchair unattended.

Switching the parking brakes on and off

1. Push the black free-wheel lever forward on both wheels.

The parking brakes are now switched off.

Attention: If the parking brakes are switched off via the free-wheel levers, the push assist can no longer be used. When these are switched on, the battery indicator will

Operate the free-wheel lever

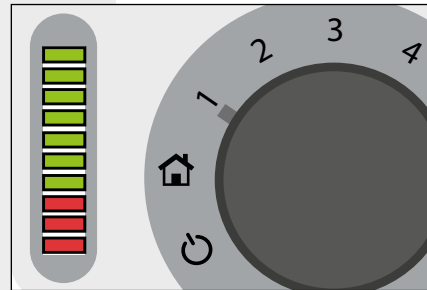
Use

Use

Checking the battery


The control panel is provided with an LED battery indicator, with which you can check the amount of charge left in the battery. If all indicator lamps are lit, the battery is fully charged.


Battery full

Attention:

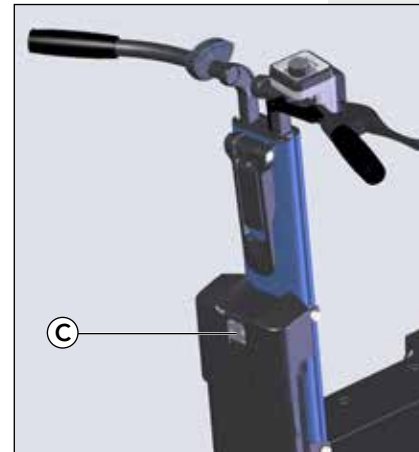- If only the bottom three (red) lamps are lit, the battery needs to be charged as soon as possible to prevent that the push assist is disabled because of an empty battery.
- If the charge is less than 10%, the last lamp will start to flash and the wheelchair's maximum speed will be limited.

Connection point battery charger

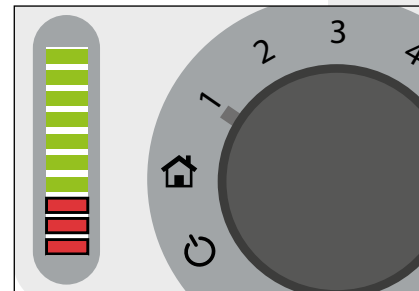Battery almost empty

- * For long-term storage of the battery: 10-30°C.
- ** If the charger lamp stays green while the battery is partially or completely discharged, there is a fault in the wheelchair or charger. The battery will not charge. In this case, disconnect the charger and please contact your dealer.

Recharging the battery

Attention:

- The position of the battery switch (on/off) does not affect charging.
- During charging you cannot use the push assist.
- Only charge the battery at ambient temperatures from +10 to +40 °C*.
- If the ambient temperature is lower than -10°C or higher than 60°C, the battery will have a shorter service life and will need to be charged more often.
- The battery needs to be fully charged at least once in every 3 months, even if the wheelchair is not used.
- The battery charger may only be used in dry areas.

The battery does not need to be empty before it is re-charged. You can re-charge the battery at any desired time. You can also interrupt the charging at any desired time. Charging a fully discharged battery takes approximately 7 hours.

1. Fix the battery charger to the connection point on the wheelchair (C).
2. Put the plug in the wall socket.
*The signal lamp on the charger will turn red**.*

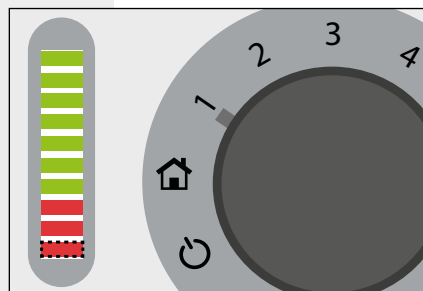
If the push assist is activated while the charger is connected, the battery indicator will display six fast-flashing lamps on the control panel as a sign that the wheelchair may not be driven.


Use

In order to drive again you should do the following:

1. Take the plug out of the wall socket.
2. Disconnect the battery charger from the wheelchair.
3. Switch the push support off and then on again.

Battery less than 10% full

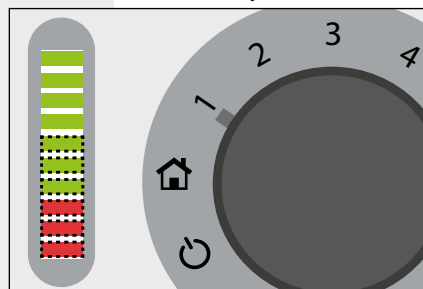Charger connected and push assist activated

Maintenance

Cleaning

During cleaning of the wheelchair the electronic components must not become wet. This concerns particularly the control on the handle, the box under the seat and the motors in the wheels. These may only be cleaned with a damp cloth.

Please see the manual concerned for permitted cleaning products for the wheelchair.

Be careful:

- As the Aura push assist is splash-proof, the wheelchair should never be sprayed with a high-pressure cleaner or hosepipe.

Tip:

- Clean the wheelchair's removable parts, such as seat, back support, arm rests etc. away from the wheelchair. This is a simple way to prevent that the control components become wet.

Minor maintenance

The push assist itself does not require any maintenance. Other maintenance is stated in the wheelchair manual.

Transport & storage

Transporting the wheelchair without occupant

The wheelchair can be folded as described in the wheelchair manual. There are a number of extra attention points regarding the push assist.

Warning:

- NEVER put the wheelchair in neutral during transport in a taxi.
- Do not confuse neutral with the parking brake. The parking brake is automatically activated when the push support is switched off.
- The automatic parking brake does NOT work if neutral is engaged.

Be careful:

- Only lift the wheelchair using fixed frame components and not removable accessories such as armrests, legrests, seating, back support etc.
- Never remove the push handles as this will damage the cabling.
- Ensure that the control unit and control lever remain free during transportation. These can become damaged if they are knocked or wedged.

Attention:

- Switch off the push assist while transporting the wheelchair without occupant.
- The Aura push assist makes the wheelchair's basic section 16 kg heavier than the standard wheelchair. With push assist, the basic section therefore weighs approximately 55 kg.

Transporting the wheelchair with occupant

The Match with Aura push assist is a 'crash-test safe wheelchair'. Thus the Match with Aura push assist can thus be transported as wheelchair with occupant in a minibus (see the indication sticker on the wheelchair).

Transport & storage

Warning:

- Transportation with occupant is only permitted if the wheelchair is provided with a crash-test safe sticker.
- Follow the instructions as indicated in the manual for the wheelchair.

Attention:

- Switch off the Aura push assist while transporting the wheelchair with occupant.
- NEVER use the auxiliary drive to drive the chair with an occupant into a minibus via the loading ramps. The angle of the loading ramps is too large for this purpose.

Warning:

- NEVER put the wheelchair in neutral during transport in a taxi.
- Do not confuse neutral with the parking brake. The parking brake is automatically activated when the push support is switched off.
- The automatic parking brake does NOT work if neutral is engaged.

Storage

The wheelchair can be folded and stored as described in the wheelchair manual. There are a number of extra attention points regarding storing the Aura push assist.

Be careful:

- The wheelchair must be stored dry. The humidity may be no more than 80%. Furthermore, temperature variations should not cause, for example, condensation to appear on the wheelchair.

Transport & storage

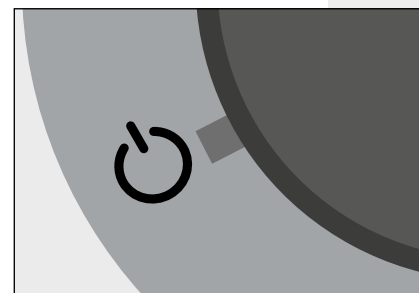
Attention:

- Environmental temperature on storage: -10 to +60°C.
- Environmental temperature on storage if the battery is connected to the charger: +10 to +40°C*.
- The battery needs to be fully charged at least once in every 3 months, even if the wheelchair is not used.**

* For long-term storage with battery:

10-30°C

** In extreme temperatures, the battery must be charged monthly.

Switch off push assist

Faults

Resetting

The Aura control performs continuous system checks to verify the correct operation of the wheelchair. In the event that a fault is found, the push assist will be disabled for safety reasons. The wheelchair's parking brake will then automatically be activated.

In the event of faults the battery indicator will start to flash. The number of flashing lamps indicates the type of fault. See the table in the 'Faults' chapter for the meaning. You can resolve many of these faults by resetting the wheelchair:

1. Switch the push assist off and wait 5 seconds.
2. Then switch on the push assist again.

If resetting does not help, or if the fault occurs often, the wheelchair should be checked by the dealer and may only be re-used if the dealer considers that the system works well.

Faults

Unusual behaviour

If the push assist's behaviour deviates from the norm, and certainly if this negatively influences the controllability of the wheelchair, the push assist may no longer be used. You should switch off the push assist. The push assist may only be re-used if the dealer has checked the system and considers that it works well.

Attention:

- Although the Aura push assist has been approved as capable of withstanding electromagnetic fields, it is possible that the Aura's working is influenced negatively by mobile and electronic equipment. In these kinds of circumstances, switch the Aura off.

Faults

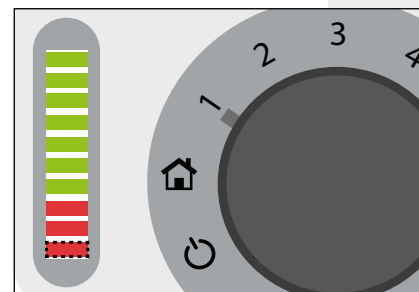
Repair

Never try to remedy faults yourself, but always contact your dealer instead. Before you alert the dealer you should first switch the system off and on again (see Resetting). If this does not remedy the fault, please contact your dealer (do NOT try to remedy the fault yourself).

Meaning of fault reports on the battery indicator

Under normal circumstances the Aura battery indicator will indicate the battery charge level. In the event of a fault, the battery indicator will start to flash and the number of flashing lamps indicates which fault has occurred.

A fault can be reset by switching the push assist off and on again after the cause of the fault has been removed.

Battery empty

Faults

Fault table battery indicator

Number of flashing indicator strips	Situation
1, fast	Poor connection with the battery or battery empty. Try to charge the battery. If this does not help, alert the dealer.
1, 1x per sec	Battery almost empty. Charge immediately.
2	Poor connection in the motor cabling. Alert the dealer.
3	Short circuit in the motor cabling. Alert the dealer.
4	Fault in the controller. Alert the dealer.
5	Fault in the controller. Alert the dealer.
6, 1x per 4 sec. short flash	Push assist in sleep mode. Switch the push assist off and then on again to end the sleep mode.
6, fast	The battery charger is connected. Switch the push assist off and then on again in order to be able to drive.
7	Control lever operated during switching on. Release the control lever and switch on again. If this does not help, alert the dealer.
8	Battery voltage is too low. Charge the battery. If this does not help, alert the dealer.
9	The brakes are set in free-wheel position. Set both free-wheel levers upwards in order to activate the brakes. Switch the push assist off and then on again in order to be able to drive.
10	Battery charge too high. Alert the dealer.

Faults

Fault table other situations

Number of flashing indicator strips	Situation
Indicator stays off even though the push assist is activated	The battery is empty. Charge the battery. If this does not help, alert the dealer.
Light bar runs from 1 to 10 lamps and back	Control lever operated during switching on. Release the control lever. If this does not help, alert the dealer.

Signal lamps on the battery charger

Lamp	Situation
Red	Battery is being charged.
Flashing red	Battery voltage is too low; discharged too deeply. The maximum charge time of 7 hours has been exceeded; charging will be suspended.
Green	Battery is full (with just a little floating charge).
Flashing green	Battery is completely full and accepts no more charge. No battery is connected.
Alternating flashing red and green	The charger alternates between charge and after-charge; this process repeats itself a number of times until the battery is completely full (light is green).

Warranty

Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product. Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The guarantee does not cover faults caused by: failure to comply with instructions for use and maintenance instructions, abnormal use, wear and tear, negligence, overloading, accidents caused by third parties, use of non original components or faults the cause of which is not related to the product.

This guarantee supersedes any other guarantee either laid down in law or verbally notified, with the exception of any written guarantee provided by Life & Mobility.

Guarantees are only valid within the EU.

Disposal of product and packaging

From an environmental perspective, we recommend that you dispose of the product and the packaging separately to as great an extent possible, allowing the used materials to be efficiently recycled.

In most cases you can hand in your used product to your wheelchair dealer. If this is not possible, you can hand in the push assist at your local waste collection station.

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This manual has been prepared with the utmost care and is based on information known to Life & Mobility at the time of publication. Life & Mobility accepts no liability for any errors in the text or the consequences thereof.

The information in this manual applies to the standard version of the product. Life & Mobility accepts no responsibility for any damage that may occur due to changes or modifications to the product after it has been sold for the first time. Neither can Life & Mobility be held liable for any damage caused by third-party work.

The information in this manual may not be reproduced and/or made public by any means without prior written permission from Life & Mobility.

Pursuant to the Dutch Trade Names Act, the trade name and trademarks used by Life & Mobility may not be considered free.

No rights can be derived from this text. Subject to changes.

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© Life & Mobility – All rights reserved