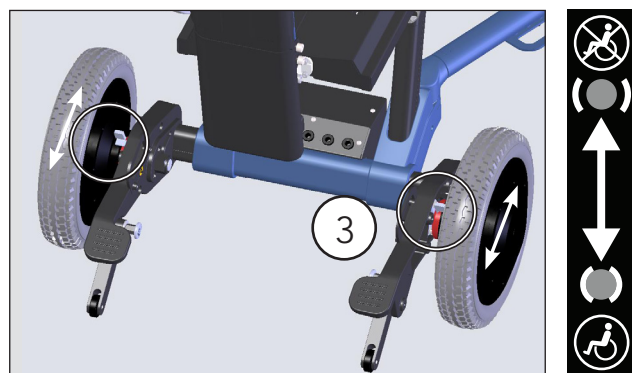
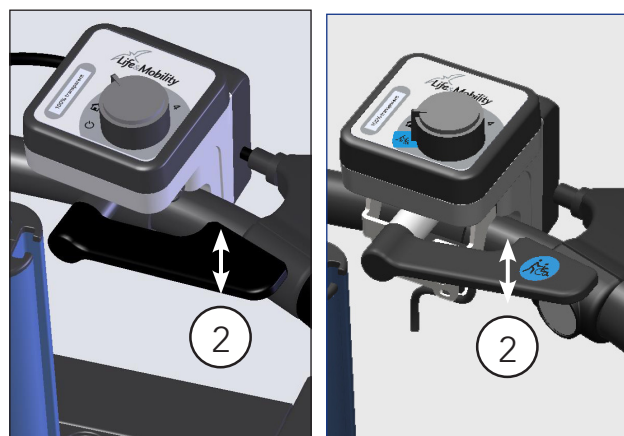
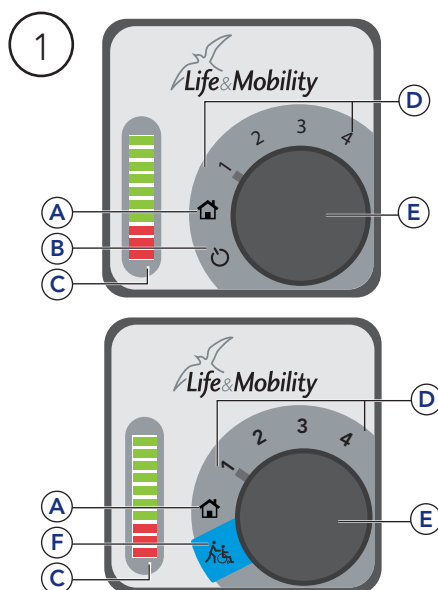
Quick start

MATCH AURA

**Vrijloopsticker / Neutral sticker / Aufkleber Leerlauf:**

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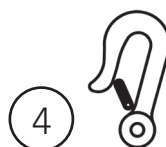
EN

Charging instructions

- Insert the charger plug into the socket on the wheelchair.
- Insert the battery charger plug into the power point.
- **Safety warning:** Never leave the battery or wheelchair unattended while charging.

Starting

- Turn the selector switch (E) to position (A).
- Speed and acceleration are limited in this "indoor mode" (A) for indoor use and manoeuvring in small spaces.

Switching off

- Turn the selector switch (E) to position (B).

Speed control

- Turn the selector switch (E) to the desired speed setting 1, 2, 3 or 4 (D).
- **Attention:** For individual speed control contact your dealer.

Moving forward with push support

- Switch on the push support (A).
- Choose the desired speed 1, 2, 3 or 4 (D).
- Depress the flat black control lever (2).
- **Attention:** When using a squeeze lever, the degree of squeezing determines the speed. The further the lever is squeezed, the higher the speed.

Reversing with push support

- To reverse in conjunction with the pinch lever, place the thumb on the flat black control levers and push down.

Braking and slowing down

- Do not depress the control lever as far, or release it completely.
- The parking brake will be activated automatically when the throttle is not operated.

In neutral / Manoeuvring without support

- Push the black neutral lever (3) on both wheels forward.
- **Attention:** When neutral has been activated push support cannot be used. If it is switched on, the battery indicator will display nine fast-flashing lamps.
- To turn off neutral, push the lever on both wheels backwards, this automatically activates the parking brake.
- Briefly turn off the Aura push support and turn it on once more to use it.

Option manoeuvre lever

- Turn the selector switch (E) to setting (F).
- In the 'manoeuvre setting', neutral can be operated without switching on support.
- Depress the flat black control lever (2). The 'click' indicates that the brakes are being unlocked.
- **Attention:** when the lever in 'manoeuvre setting' is not being operated, the system is automatically on the parking brake.

Transport in a taxi (4)

Warning:

- Transportation with occupant is only permitted if the wheelchair is provided with a crash-test safe sticker.
- NEVER put the wheelchair in neutral during transport in a taxi and turn off the push support.

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