CUMULIX

KELVIN

GEBRUIKERSHANDLEIDING  
USER MANUAL  
MODE D'EMPLOI  
GEBRAUCHSANWEISUNG

Inspired by life

 Life & Mobility

## Inhoud

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## Contents

Foreword	4
Product description	5
Optional	6
Technical specifications	7
Safety instructions	10
Settings & adjustments	13
Use	17
Transport & storage	19
Maintenance	20
Technical problems	21
Warranty	23
Used wheelchairs / environment	24

## Foreword

The Cumulix Kelvin Classic is a tilt-in-space wheelchair that has been specially designed for permanent wheelchair users who need a wheelchair with maximum comfort. The Cumulix always provides the care professional with a customized personal solution, irrespective of the weight, body dimensions and state of health of the passive user.

The Cumulix has two gas springs which give the wheelchair a stable, comfortable tilting frame which, in addition to a large tilting range, is light to operate with or without a user. The backrest angle is adjustable.

About Cumulix Kelvin Classic:

- Available in 3 width sizes.
- Suitable for both indoor and outdoor use.
- The maximum user weight is 125 kg. This is indicated on the CE sticker (X).
- Complies with the European wheelchair standards EN12182 & EN12183.

The Cumulix therefore complies with the mandatorily applicable European REGULATION (EU) 2017/745 for Medical Devices.

**Important:**

- Please read these instructions for use carefully before using the Cumulix for the first time.
- The optional extras for the Kelvin seat elements are described in the Kelvin specific seat element manual.
- Paragraphs in this document that are relevant to your safety are marked with a safety symbol.

CE sticker

## Product description

Product description

**Checking the wheelchair on delivery**

Before you start using it, check that the wheelchair delivered, matches the product description below. If you notice any damage caused during transit, please inform your dealer. Life & Mobility trusts that the Cumulix will meet your expectations.

**Product description**

1. Headrest (option)
2. Push handle
3. Park setting
4. Brake lever
5. Operating lever for seat angle
6. Kelvin Classic seat element
7. Armrest (option)
8. Rear wheel
9. Front wheel
10. Anti-tip supports\*
11. Step tube

Not illustrated:

- User manual

\* Anti-tip supports for preventive safety are supplied as a standard feature with the Cumulix

## Optional

In addition to the standard components for the Cumulix, the following optional components are also available.

### Headrest

- Standard headrest
- Kelvin headrest small
- Kelvin headrest big
- Kelvin headrest small with Edison mechanism
- Granular filled neck cushion

### Armrests

- Height-adjustable armrest wide
- Flex arm support

### Backrests

- Lumbar support

### Brakes

- Drum brakes operated by accompanying person


### Worktop

- Upholstered worktop

### Accessories

- Terry cover
- Refill packet granule-latex
- Seat belt

## Technical specifications

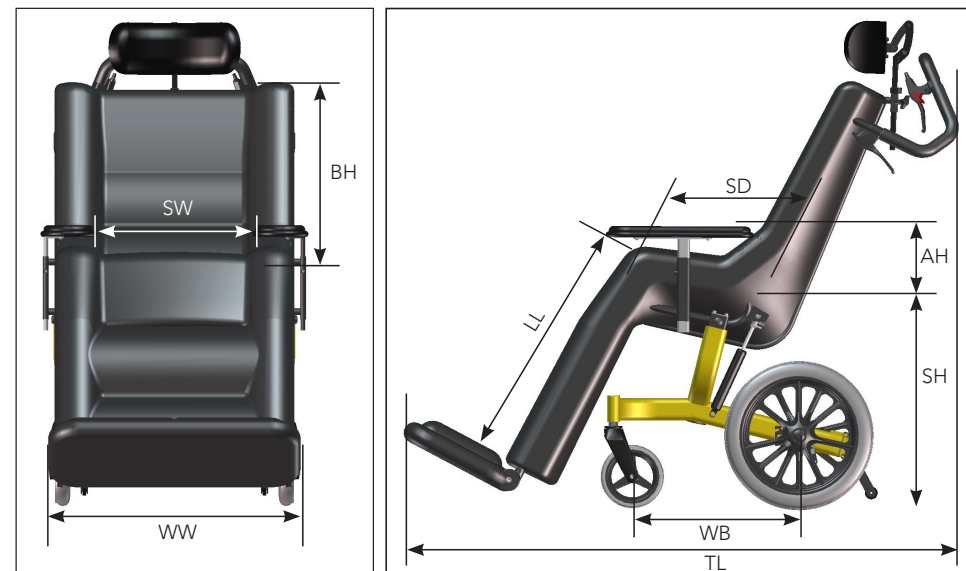Abbrev.	Description	Measurements
SW	Seat width*: Size S Size M Size L	34 - 40 cm 40 - 46 cm 46 - 52 cm
SH	Seat height	60 cm
SD	Seat depth for all 3 sizes*	48 cm
BH	Back height	60 cm
AH	Armrest height	17 - 26 cm
LL	Lower leg length	35 - 50 cm
-	Seat angle adjustment measured under load	10° - 40° (range is adjustable)
-	Backrest angle adjustment	85° - 140° (range is adjustable)
WB	Wheel base	46 cm
WW	Wheelchair width	70 cm
TL	<b>Attention:</b> The length of the total combination is larger than the recommended dimensions.	
-	Total weight basic model	39 kg
-	Maximum user weight	125 kg
-	Conforms to	EN 12182 EN 12183
-	Recommended ambient temp.	Between -5°C and +40°C.

\* The minimum size is measured on the seat cushion. Adding more or less filling to the Kelvin cushions allows the seat width and depth to be individually adjusted.

## Technical specifications

Disclosure information (ISO)							
Standard reference		min.	max.	Standard reference		min.	max.
ISO 7176-5	Overall length with legrest		1450 mm	ISO 7176-7	Seat plane angle	10°	40°
ISO 7176-5	Overall width	700 mm	700 mm	ISO 7176-7	Effective seat depth	480 mm	480 mm
N.A.	Folded length			ISO 7176-7	Effective seat width	340 mm	520 mm
N.A.	Folded width			ISO 7176-7	Seat surface height at front edge	600 mm	600 mm
N.A.	Folded height			ISO 7176-7	Backrest angle	85°	140°
ISO 7176-5	Total mass	39 kg	39 kg	ISO 7176-7	Backrest height	600 mm	600 mm
ISO 7176-5	Mass of the heaviest part	39 kg	39 kg	ISO 7176-7	Footrest to seat distance	350 mm	500 mm
EN12183	Static stability downhill		10°	ISO 7176-7	Leg to seat surface angle	90°	180°
EN12183	Static stability uphill		10°	ISO 7176-7	Armrest to seat distance	170 mm	260 mm
EN12183	Static stability sideways (Left & Right)		20°	ISO 7176-7	Front location of armrest structure	540 mm	540 mm
N.A.	Energy consumption			N.A.	Handrim diameter		
N.A.	Dynamic stability uphill			ISO 7176-7	Horizontal location of axle	-100 mm	-100 mm
N.A.	Obstacle climbing			ISO 7176-5	Minimum turning radius	2100 mm	
N.A.	Maximum speed forward						
N.A.	Minimum braking distance from max speed						

## Technical specifications

- The wheelchair conforms to the following standards:
- a) Requirements and test methods for static, impact and fatigue strengths (ISO7176-8)
  - b) Requirements for resistance to ignition in accordance with ISO 7176-16

### Expected service life

Life & Mobility assigns a service life of 7 years to the Cumulix. This means that during this period, the product is fixable, safe and usable.

The statement above is only applicable if the user uses the product in the manner that Life & Mobility intended, marked with CE and including any original accessories.

## Safety instructions

### General

#### **Be careful:**

- Never change the construction of the wheelchair.
- Use only original Life & Mobility parts and accessories. These are tested and safe.

#### **Attention:**

- Directions, safety symbols and instructions are attached to this wheelchair. They must not be covered over or removed. They must remain attached to the wheelchair throughout its lifespan and be clearly legible.

### Assembly and adjustment

#### **Warning:**

- Before using the wheelchair, check that all its parts are properly mounted and secured.

#### **Attention:**

- Ensure that the armrests, the lower leg length and the headrest, if applicable, are adjusted to the correct height.

### Sitting in the wheelchair

#### **Warning:**

- Make sure that the occupant's fingers and limbs are not trapped between the frame and the wheels or between the wheels' spokes. Always be very careful when:
  - The wheelchair is tilted far backwards
  - The occupant is sitting lopsided in the wheelchair.
  - The occupant has his/her arms hanging limp over the armrest.
  - A combination of the above occurs.

## Safety instructions

### Use of the wheelchair

#### **Attention:**

- Set the brake on the wheelchair before making a transfer.
- The Cumulix is not intended to carry additional loads, do not place luggage on, or hang bags etc. from the push handle or elsewhere. The balancepoint may change and the wheelchair can tip over more easily.
- When using the chair on a slope, the centre of gravity will change and the wheelchair can tip over backwards more easily.
- Accompanying persons must never leave the wheelchair on a slope without engaging the brake.

### Slopes and other obstacles

#### **Warning:**

- Never drive on a slope at an angle exceeding 5 degrees.
- Do not drive on slopes when the seat is tilted.
- When driving up a slope, the centre of gravity will shift and the wheelchair can tip backwards more easily than on level surface.
- When driving down a slope, the centre of gravity will shift and the occupant can tip forward.
- Do not turn around on a steep slope. If the wheelchair is crosswise on a steep slope, the risk of tipping is significant.
- Always go off a pavement or small step on steps backwards so the wheelchair cannot tip forward and to prevent the user from falling out of the chair.
- Assistants must never leave the wheelchair on a slope without applying the brakes.

## Safety instructions

### Attention:

- When going down a slope, always keep your speed under control. If necessary, brake the wheelchair using the brake levers.

### Miscellaneous

### Warning:

- Keep the wheelchair away from naked flames.
- Do not leave the wheelchair in direct sunlight: the upholstery may get too hot, which can burn the skin or make it hypersensitive.

### Be careful:


- Do not allow the wheelchair to come in contact with seawater as seawater is corrosive and will rust the wheelchair.
- Do not allow the wheelchair to come in contact with sand, as it may penetrate into the moving parts of the wheelchair, causing unnecessarily fast wear and tear.
- Never use abrasive or aggressive cleaning products or solvents such as thinner, benzene, etc., to clean the wheelchair.
- Tightening the adjustment screw too far can damage the wheelchair.

## Settings & adjustments

Life & Mobility recommends that you consult your dealer with regard to the settings. You can easily make adjustments yourself.

Use the following tightening guidelines:


- M5 socket screws = 6 Nm
- M6 socket screws = 10 Nm
- M8 socket screws = 25 Nm
- M10 socket screws = 50 Nm

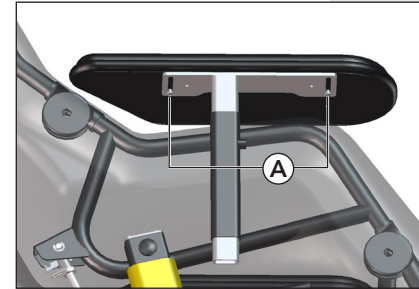Adjusting the seat width

Adjusting the backrest angle

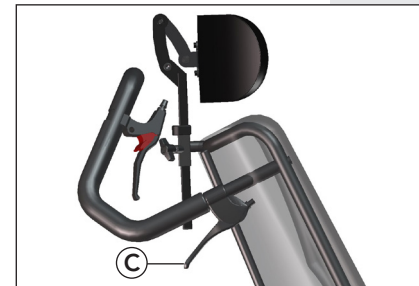Adjusting the seat angle

### Frame

#### Seat width

It is possible to fit the armrests, further in or out. This adjustment can be made separately on the left and on the right using the screws (A).

#### Backrest angle (one-time adjustment)

1. Loosen the socket screws (B) in the backrest hinge points.
2. Set the backrest to the required angle.
3. Retighten the socket screws.

#### Seat angle

The seat angle can be adjusted using the 2 gas springs and the operating lever (C) near the push-handle.

1. Press the operating levers to disengage the gas springs.
2. Adjust the seat angle as required.
3. Release the operating levers.

#### Seat angle range

The seat angle range is 30°. It can be adjusted from 10° to 40°.

## Settings & adjustments

### Anti-tip supports

The anti-tip supports can be adjusted using the pull knob (D).

There are 2 settings: activated and folded away.

The anti-tip support - if properly adjusted - protects the wheelchair user from tipping backwards. The risk of tipping the wheelchair backwards is influenced by:

- the position of the user's center of gravity in the wheelchair (which is determined by factors such as axle position, backrest angle, seat angle, seat tilt, and weight distribution over the body (e.g., amputations)).
- any restlessness or spasms of the user.
- the slope on which you are driving.

Adjust the anti-tip as necessary based on the above considerations.

### Armrests

The height and width of the armrests can be adjusted.

#### Adjusting the height

1. Loosen the socket screw (E) a few turns.
2. Set the armrest to the required height.
3. Retighten the socket screw.

#### Adjusting the width

1. Loosen the screws (F) fully.
2. Set the armrest to the required width.
3. Retighten the screws.


Anti-tip supports activated

Anti-tip supports folded away

Adjusting the armrests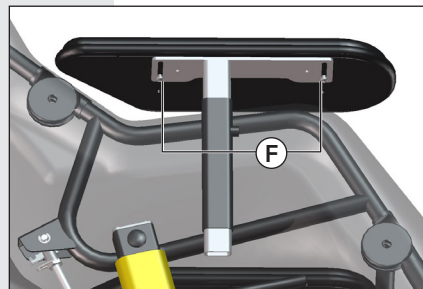

Adjusting the armrests

## Settings & adjustments

### Legrests

#### Lower leg length

1. Loosen the M8 socket screw (A) one turn on both sides. Socket screw (A) is located underneath the zip fastener of the seat cover.
2. Set the legrest to the required lower leg length.

#### Attention:

- Make sure the legrest is at least 5 cm off the floor.


3. Retighten the socket screw.

#### Setting the knee angle


1. Loosen the socket screw (B) one turn on both sides.
2. Set the knee angle to the desired position.
3. Retighten the socket screws on both sides.

#### Setting the ankle hinge / foot rest


1. Loosen the socket screw (C) one turn on both sides.
2. Set the ankle angle to the desired position.
3. Retighten the socket screws on both sides.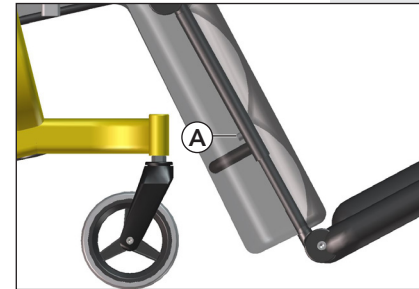

Adjusting the lower leg length

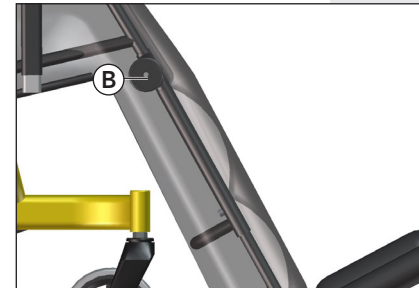Adjusting the knee angle

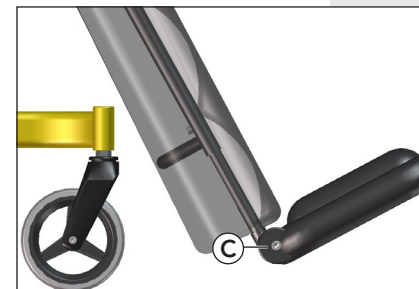Setting the ankle hinge / foot rest

## Settings & adjustments

### Headrest (optional)

Removing / adjusting the height of the headrest\*

1. Loosen the wingnut (A).
2. Adjust the height and retighten the wingnut, or remove the headrest from the wheelchair.

\* If applicable

Adjusting the headrest height

### Lumbar support (optional)

Place the lumbar support behind the backrest upholstery as shown in the image on the right.

**Tip:**

- This is an indication of a possible position, using the vertical velcro straps, the lumbar support can be positioned as desired.

A Backrest  
B Seat

Positioning lumbar support


### Seat belt (optional)

The seat belt is intended to keep the user in position. Attach it on or around the metal seat frame.

**N.B.:**


- The optional wheelchair seat belt is not suitable for use as a safety belt.

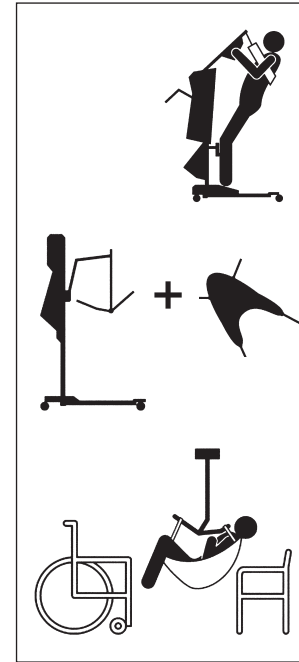Making a transfer

Going up a sidewalk, step, etc.

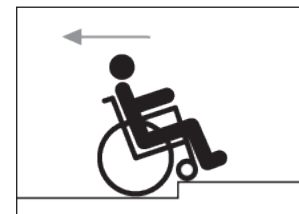Leaving the sidewalk, step, etc.

### Transferring to and from the chair

The Cumulix is a wheelchair for users with various handicaps. Transfers from the wheelchair to, for example, a bed, toilet or shower and vice versa can be made in various ways. The extent to which the wheelchair user is able to cooperate determines how he or she should be lifted. Both active and passive transfers are possible with the Cumulix, with the aid of the care provider or using a lift.

**Attention:**

- Always engage the brake before transferring to or from the chair.

Forwards transfer

**Tip:**

- During a transfer from a passive lift into a Cumulix, first tip the wheelchair without user so that the user can be optimally positioned in the chair.

### Tackling obstacles

1. Press the push-handles downwards and push with your foot on the step tube.
2. Push the wheelchair onto the obstacle and remove your foot from the step tube.
3. Push the wheelchair further and lift the back wheels over the obstacle by the push bar.

**Attention:**

- You can go up a sidewalk or small step forwards but it is better to leave the sidewalk or step backwards.
- Keep your speed under control on gentle slopes.

## Use

### Brakes

The brakes of the Cumulix are correctly adjusted ex factory. Your Cumulix has drum brakes with a parking option.

#### Setting the parking option


1. Squeeze brake lever B.
2. Pull the red parking option knob towards you on both sides until latched.

*When the parking brake is engaged, the brake lever remains pressed in.*


#### Adjusting the brakes

It may be that over the years the braking power decreases or is no longer uniform for both wheels. You should then adjust the brakes.


1. Loosen the locknut (C).
2. Turn the adjustment bolt (D) outwards\*.
3. Check that the braking power is adequate.
4. Adjust it again if necessary.
5. Tighten the locknut (C).
6. Repeat this on the other side, if required.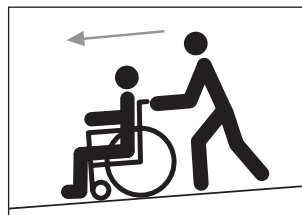

Gentle slope

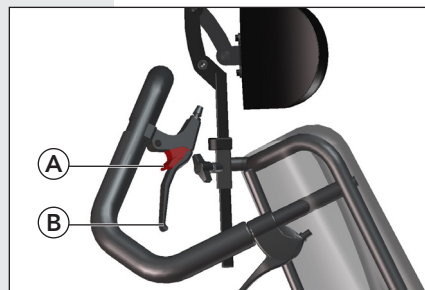Brakes

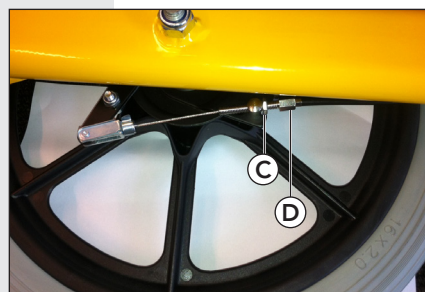Adjusting the brakes

## Transport & storage

### Taxibus

The Cumulix has NOT been tested by the TNO as 'crash test-proof wheelchair' in compliance with ISO 7176-19. The Cumulix can therefore NOT be transported, with user, in a taxibus.

### (Long term) storage

The Cumulix cannot be folded. To assist in storage:

- Put the wheelchair in an upright position.
- Fold away the foot rest.
- Fold back the legrest.

For instructions go to chapter 'Settings and adjustments', paragraph 'Legrests'.

## Maintenance

### Minor maintenance (every two weeks)

You can do minor maintenance on the Cumulix yourself. Clean your wheelchair with a damp cloth, water and an all purpose cleaner.

#### Attention:

- Never use abrasive and aggressive cleaners or solvents (thinners, benzine or similar).

The gas springs and the gas spring block (the rectangular block at the end of the gas spring) should be lubricated regularly with, for example, Teflon spray. This will keep them working properly.

Check that all fastenings are secure. Tighten if necessary.

#### Attention:

- Over-tightening the screws can damage the wheelchair.

### Major maintenance (annual)

Your dealer will be happy to carry out the major maintenance of your wheelchair. You can then be sure that you will be getting the best possible use out of your wheelchair. You should also consult your dealer if components need replacing.

## Technical problems

In the unlikely event that you experience technical problems with your wheelchair, please contact your dealer. The dealer will decide whether he can repair the chair or whether it will have to be returned to the factory.

You may be able to solve some problems yourself.

*The seat angle can no longer be adjusted*

- Check whether the operating cable is bent or has become trapped somewhere.
- Check whether the gas spring block moves when the handle is pressed. Lubricate the gas spring block if necessary.

*Movable components such as the height-adjustable armrest, legrest get stuck*

- These components can become jammed in the frame if too much pressure is exerted on them. Carefully tap the components loose with a plastic hammer.

*Socket screws for one-off adjustment cannot be loosened*

- The screws may have been over-tightened. Try placing a tube over the long end of the Allen key so as to lengthen the turning arm. Now try loosening the screw.
- The socket head may be stripped: Try gripping the screw with a monkey wrench. If this fails consult your dealer.

*The brakes have insufficient braking power or do not brake uniformly on both sides*

- You should adjust the brakes; go to page 18.

## Technical problems

### Attention:

- *All repairs other than those described above must always be performed by an authorised dealer. Improper repairs may endanger your health. Furthermore, repairs carried out by unauthorised persons will invalidate the guarantee on your chair. Use your common sense and decide whether you are in a position to resolve technical problems on the wheelchair yourself. If in doubt, always consult your dealer. When adjusting and installing or removing parts, consult this manual and follow your dealer's advice. Do not place yourself or the wheelchair user in danger.*

### Recalls

In case of a product recall, this will either be mentioned on the **[www.life-mobility.com](http://www.life-mobility.com)** website or you will be personally informed by your dealer.

## Warranty

### Terms and Conditions of Warranty

The Life & Mobility products are made with care and are carefully inspected before leaving the factory. Should it emerge that a product does not meet expectations please contact the party where you purchased the product.

Life & Mobility respects the laws and regulations that protect the consumer and that apply in the country in which the product is purchased.

The warranty does not apply to defects caused by: failure to observe the instructions for use and maintenance, use other than normal use, wear and tear, negligence, overloading, third-party accidents, non-original parts used and defects not caused by the product.

This warranty replaces any other warranty, either stipulated by law or communicated verbally, except for anything guaranteed in writing by Life & Mobility.

Warranties are only valid within the EU.

## Used wheelchairs / environment

### **Disposal of the product and packaging**

From the environmental viewpoint, we recommend disposing of the packaging and the wheelchair in such a way that the materials used can be efficiently recycled.

In the majority of cases, you can hand in your used wheelchair to your wheelchair dealer. If this is not possible, you can hand in the wheelchair at your local refuse collection location.

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